Enjoy Mobility

K-MOD™

KNEE-MODULAR SOLUTION

TECNICA OPERATORIA 3REF
3REF OPERATING TECHNIQUE
TÉCNICA QUIRÚRGICA 3REF

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The K-MOD system is a synthesis of innovations in total knee replacement that have proven their clinical efficacy. It's a complete modular system thanks to a suite of possibilities designed to satisfy the surgeon's approach to each patient.

The fixed insert models come in versions for conservation of the posterior crossed ligament (CR), UltraCongruent (UC), the development of the medial pivot concept: Dynamic Congruence (DC) and posterior stabilized (PS FXD). In the mobile insert version, the implants come as an UltraCongruent rotating platform (UC MOB) and posterior stabilized (PS MOB).

All the femoral and tibial designs are available in cemented, uncemented and anallergic versions; all the articular surfaces, femoral and tibial, are mirror polished. The system comes complete with a patellar component and additional stems for the tibial component.

The instrument set is modular, compact and easy to use. The operating technique starts with the creation of the extension gap, thus allowing the surgeon to begin with the tibial or femoral distal approach; alternatively, the surgeon can proceed with all the femoral resection and then the tibial one. The femoral reference system is defined as "3REF", as it allows the use of an anterior, posterior, or hybrid reference, starting from both, moving the resection block at millimetric intervals. Optional additional instruments: FLEXION SPACER BLOCK or TENSOR for perfect ligament balancing. Kinematic alignment (KA EXTREME). Reconstruction of the joint line in case of revision of unicompartmental prosthesis (UNI REV).

Technologies are available for personalized 3D preoperative planning through online platforms for a completely extra-medullary approach using navigation or inertial sensors or augmented reality. Solutions that meet the specific needs of each patient, minimizing complications such as excessive wear, instability and the risk of patellar dislocation.

Thanks to full modularity with K-MOD REV and K-MOD REV CCK revision implants, the system makes it possible to deal with a wide range of situations, being designed to provide less invasivity, bone sparing, proper alignment, balancing of the extension/flexion gaps and the restoration of knee biomechanics.

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On the patient's most recent X-rays, perform thorough pre-op planning on the AP and ML planes by using the X-Ray templates K-MOD: K-MOD X-Ray Template Fem AP/ML, Tib AP/ML, Fix/Mob, Stem (KMOD-XR-000÷008; KMODPS-XR-000÷003).

Evaluate (magnification: 15%): sizes of the femoral and tibial components, joint line, diameter and length of the optional tibial stem if needed.

Evaluation is indicative and must always be checked and modified during surgery if necessary.

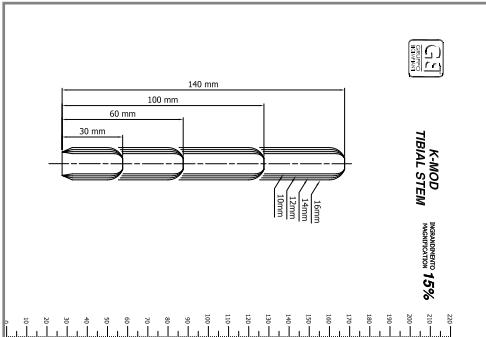
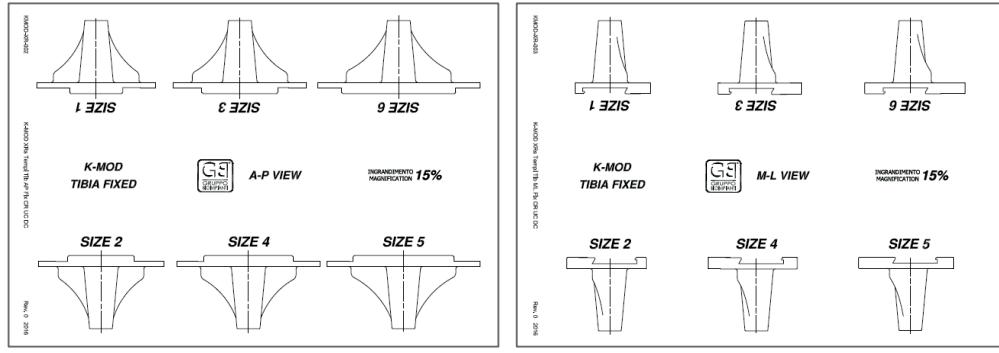
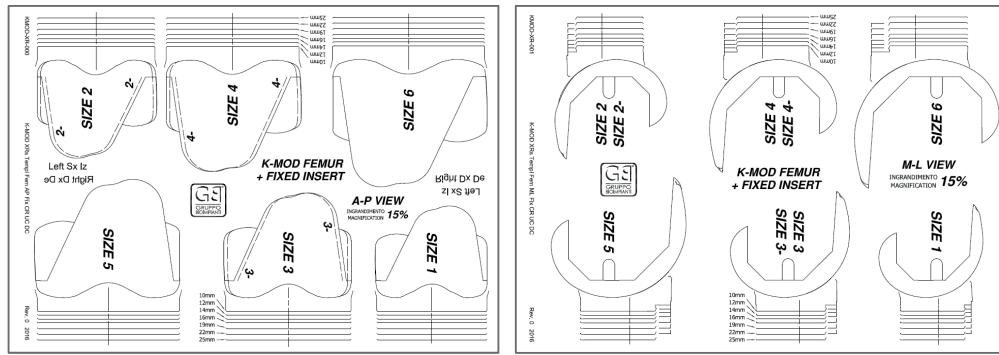
Technologies are available for personalized 3D preoperative planning through online platforms.

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To simplify the nurse's intra-op tasks, the instrumentation develops from a minimum of only four trays with the following logical sequence:

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PIN

The instrument set includes different types of pin. Unless specified otherwise, preferably use the Pin Dia 3,2 X 80mm (300110144).

Preferably insert the pin with the motor instead of the hammer, using the appropriate Jacobs Adaptor (300110377). Always proceed by inserting the pin into the hole of the cutting guide by hand, even until contact with the bone, and then proceed to penetration into the bone, taking care to keep in line.

We suggest having around 6/8 pins available.

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Perform access to the knee joint based on the approach deemed appropriate by the surgeon.

The K-MOD TOUCH surgical technique preferably develops from the extension gap and allows starting with the proximal tibial or femoral distal resection, according to the operator surgeon's preference.

In the approach, consider the following features and possibilities:

- it is possible to perform proximal tibial resection in an extra or intramedullary way;
- in case of Posterior Cruciate Ligament sacrifice or use of an additional tibial stem, do not exceed the posterior inclination of the tibial resection (0-3 °);
- technologies are available for a femoral extra-medullary approach;
- the references for the femoral measurement are, optionally: anterior, posterior, hybrid;
- it is possible to perform the final verification of the most suitable insert even with the already implanted femoral and tibial components
- the system is completely compatible in all its versions: CR, UC, DC and PS.

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INSTRUMENT PRE-SETTING RECOMMENDATIONS. These indications reflect common surgical practice. They are intended not to be binding but to facilitate the pre-setting of certain instruments, especially for the instrumentation nurse.

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Make a longitudinal incision of the quadriceps to facilitate eversion of the knee cap and achieve adequate knee flexion.

The extra-medullary tibial resection guide consists of four parts: the Spikeless Tibial Resection Proximal Guide (300110115), the appropriate Tibial Cutting Block (RIGHT 300110681, LEFT 300110680), the Tibial Distal Guide (300110122) and the Ankle Clamp (300110133).

The guide must be assembled before being applied.

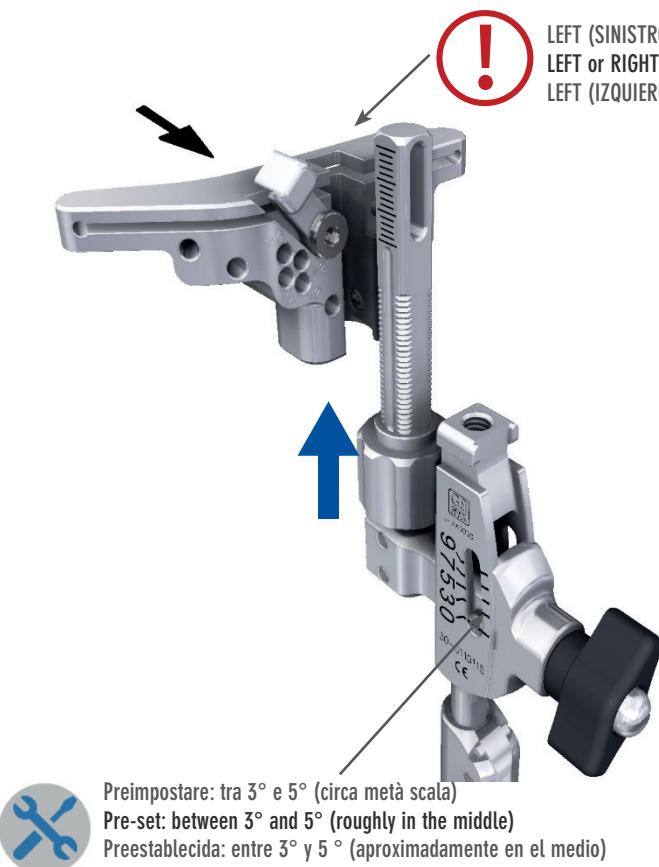
For the intra-medullary version, see the section entitled OPTION: INTRAMEDULLARY TIBIAL RESECTION.

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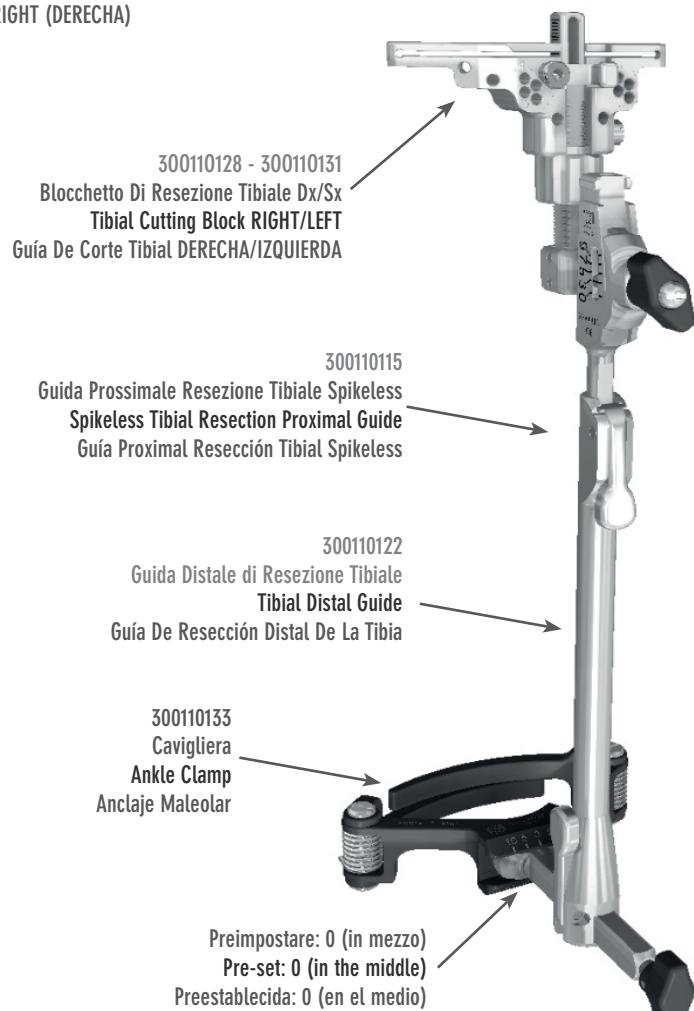
Make sure you have chosen the correct Tibial Cutting Block (LEFT, RIGHT). Preselect the desired tibial slope value (range: 0°-9°) on the Spikeless Tibial Resection Proximal Guide.

Enclose the malleoli with the Ankle Clamp and adjust the length of the guide so that the Tibial Cutting Block is at roughly the desired cut height. Align the tibial resection guide on the proximal tibia with the third medial of the tibial tuberosity and/or distally with the second metatarsophalangeal bone. In the sagittal plane, the guide must be parallel to the anterior tibial cortex.

Lock it in this position using a temporary Pin 3,2mm x 80mm (300110144) in the middle of the proximal slot. The pin will in any case make it possible to correct the posterior slope, the varus-valgus mechanical axis and the resection height.

The locking pin may also be inserted in the slot of the Tibial Cutting Block.

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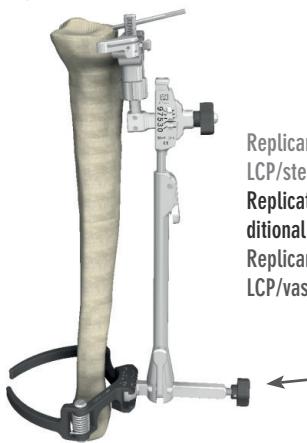
To use the entire slot of the Tibial Cutting Block, place it on the upper edge of the guide

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The alignment of the frontal mechanical axis in the varus-valgus plane may be adjusted by unscrewing the knob on the Tibial Distal Guide and sliding the guide on the Ankle Clamp in the ML plane. The posterior slope may be adjusted by using the Angel Wing (300110139) and operating the knob on the Spikeless Tibial Resection Proximal Guide and/or the AP trigger on the Tibial Distal Guide. In general, proximal tibial resection should follow the tibia's natural posterior slope.

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

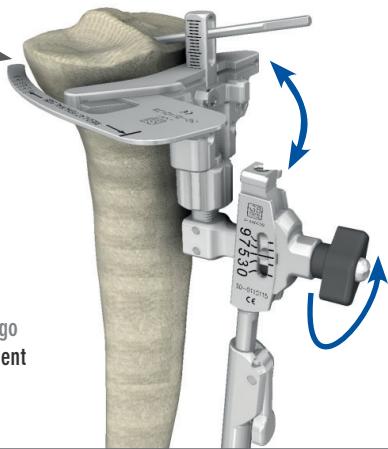
Position the Tibial Stylus (300110140) in the slot on the Cutting Block. The surgeon may choose to use as a cutting reference the lowest point on the most worn side of the tibial plateau and carry out a minimum resection (2mm tip for 2mm resection) or the highest point on the least worn side (10mm tip for 10mm resection). Check that the cut is adequate and if necessary modify the size of the resection by operating the ring and making a micrometric vertical shift of the Tibial Cutting Block.

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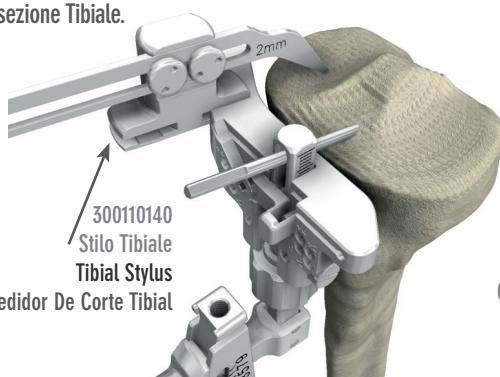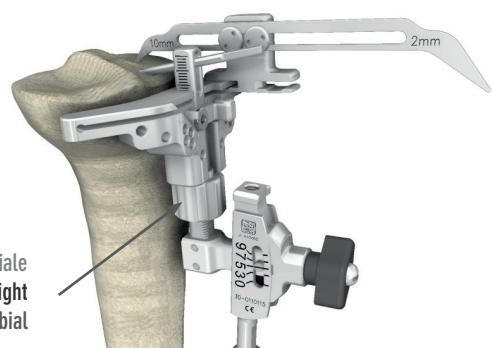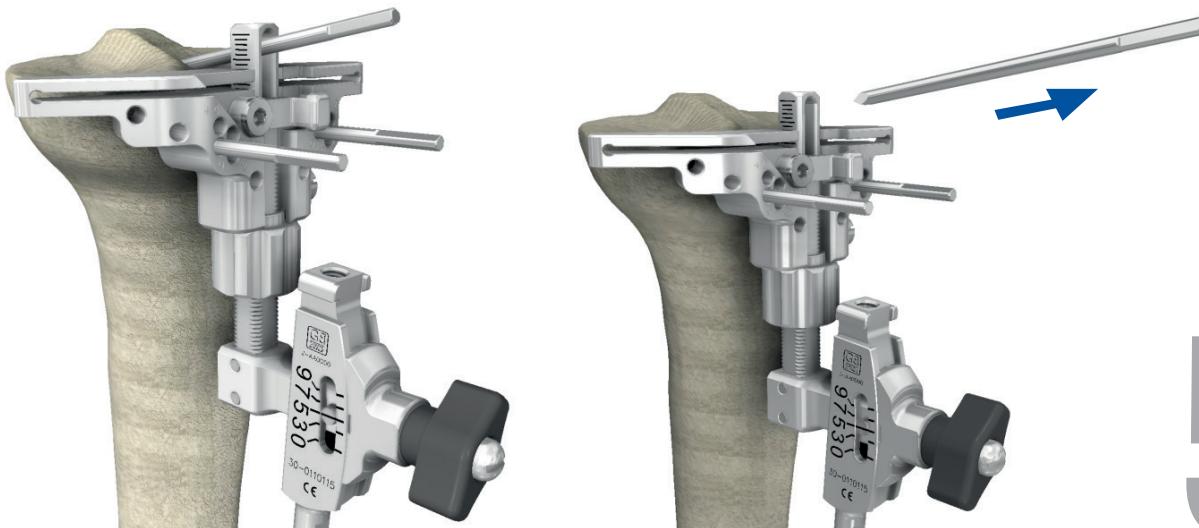
Anchor the Cutting Block with two Pin 3,2mm x 80mm (300110144) in the parallel 0 rows of holes.

Remove the temporary pin from the slot on the Spikeless Tibial Resection Proximal Guide.

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The guide can be made more stable by inserting a third pin in an oblique hole on the medial side of the Cutting Block. Such pin may be either the Pin 3,2mm x 80mm (300110144), the Pin with stop 3,2mm x 40mm (300110198) or the Threaded pin Ø3,2mm- Ø4,2mm (300110379).

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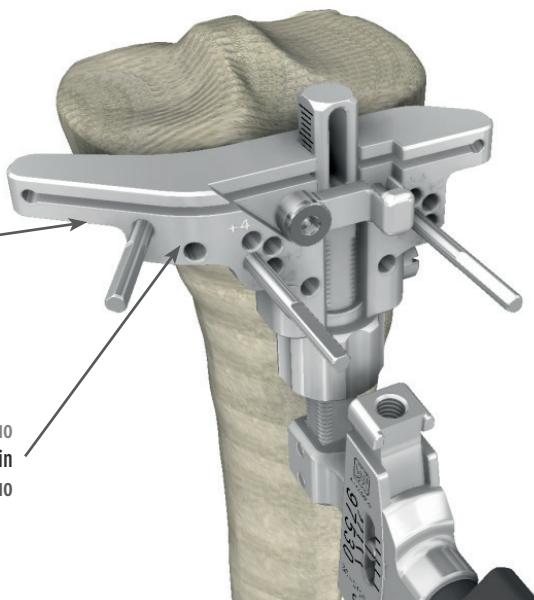
In case of a +4mm RECUT, just use the more medial oblique hole

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Make sure that the trigger on the Cutting Block is properly closed and perform the resection of the proximal tibia keeping the extra-medullary tibial resection guide in place. Use the Angel Wing to check the planarity of the cut and, if necessary, refinish the cut.

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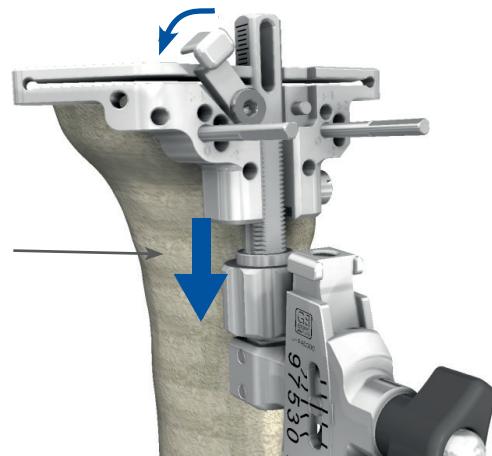
If an oblique pin has been inserted, remove it. Raise the trigger on the Cutting Block and slide the ring downwards. Remove the extra-medullary tibial resection guide. Remove the Cutting Block by sliding it outwards. Leave the two parallel pins in for the next steps.

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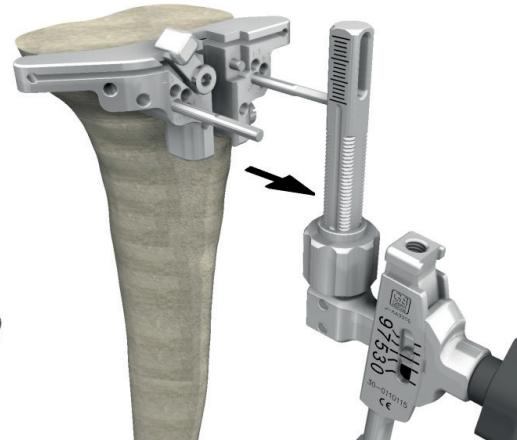
Use the Femoral IM Reamer 8.25 mm (300110145) to prepare the intramedullary hole in the femoral canal. The drill hole must be located approximately 1 cm above the femoral attachment of the posterior cruciate ligament and slightly medial to the mid-line of the femur.

Removal of osteophytes from the margins of the intercondylar notch may aid identification of landmarks.

Insert the appropriate IM Rod (LONG 300110150; SHORT 300110376) into the femoral canal hole using the relative T-Handle For IM Rod (300110151). The IM Rod has longitudinal grooves that limit pressurization of the canal and the risk of emboli.

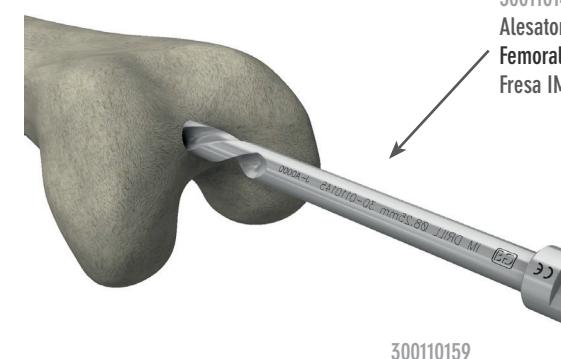
Remove the T-Handle.

Assemble the K-MOD 3REF Femoral Sizer Extra-Rot (300110702), the Distal Femur Cutting Block (300110159), with the DISTAL FEMUR marking visible on top and the K-MOD 3REF Femoral Valgus Guide (300110701), with the FRONT marking visible on top.

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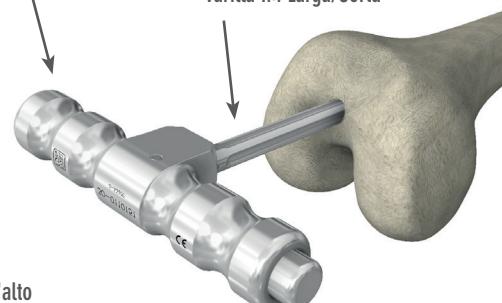
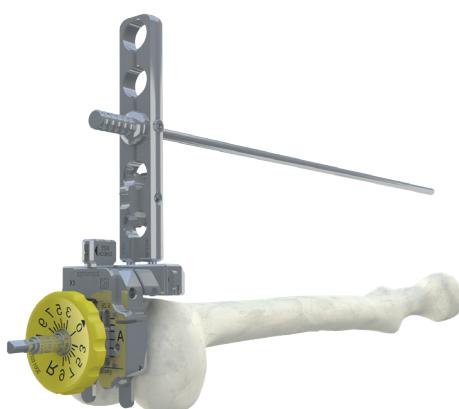
Preselect the valgus angle value determined in pre-operative planning: turn the bushing so that the marking relative to the leg being operated on, L (Left) or R (Right), is visible on the side facing upwards and set the desired valgus angle (range: 0° - 9°) by positioning the relative marking by the sign on the hole through the 3REF Femoral Valgus Guide. Insert the Distal Femur Cutting Guide on the IM Rod making sure that the Femoral Sizer Extra-Rot is in contact with at least one of the two femoral condyles.

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It's possible to check proper alignment on the mechanical axis by inserting the Handle (300110682) into the holes of the Distal Femur Cutting Block and the Alignment Rod (300110161) in the Handle, making sure it's well tightened so the Rod can't shift.

If necessary, correct the pre-set valgus angle by turning the bushing, again with respect to the marking relative to the leg being operated, L (Left) or R (Right)

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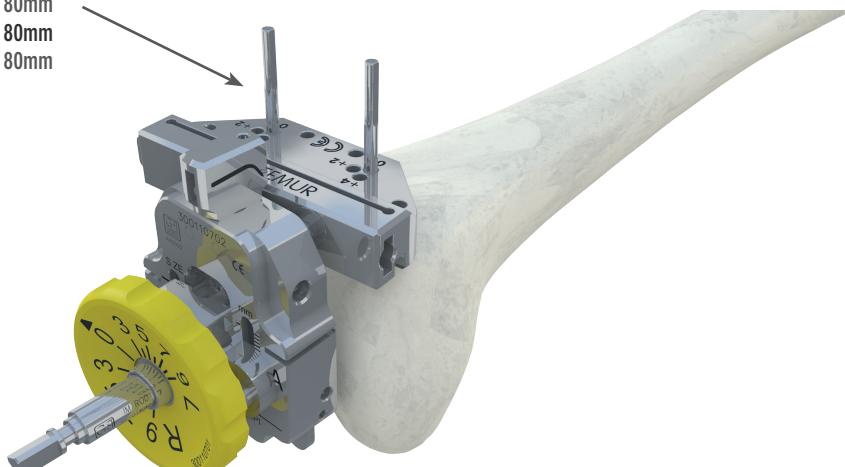
300110144
Pin 3.2mm x 80mm
Pin 3.2mm x 80mm
Pin 3.2mm x 80mm

Fix the Distal Femur Cutting Block with two Pin 3.2 mm x 80mm (300110144) in the parallel rows of 0 holes.

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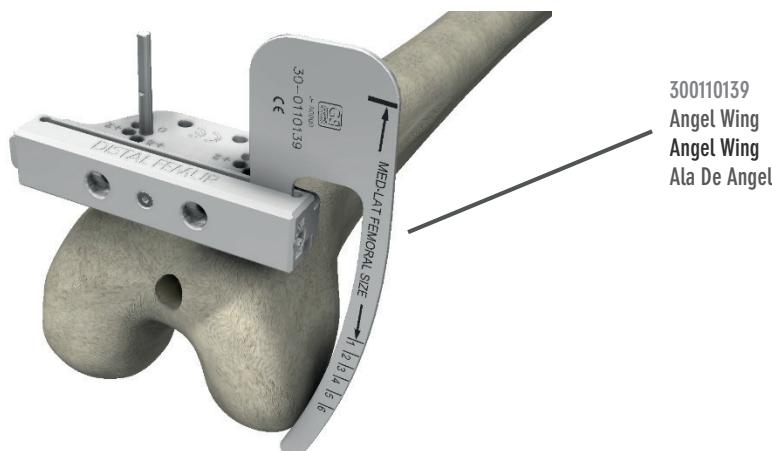
Release the Guides by pressing the button on the side of the Distal Femur Cutting Block and slide them off the IM Rod. Remove the IM Rod from the femoral canal using the relative T-Handle.

Check the position of the resection plane using the Angel Wing (300110139). If desired, 2 mm or 4 mm of additional bone may be resected by repositioning the Cutting Block over the Pins through the +2 or +4 holes respectively, or 2 mm less bone may be resected using the -2 holes.

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
The stability of the Cutting Block can be improved by inserting one or two Pins in the oblique holes in the Cutting Block. The following pins are available: Pin 3.2mm x 80mm (300110144), Pin With Stop 3.2mm x 40mm (300110198), Threaded Pin Ø3.2mm-Ø4,2mm (300110379).

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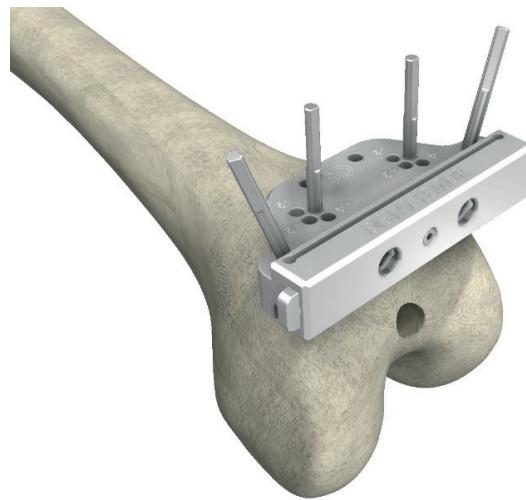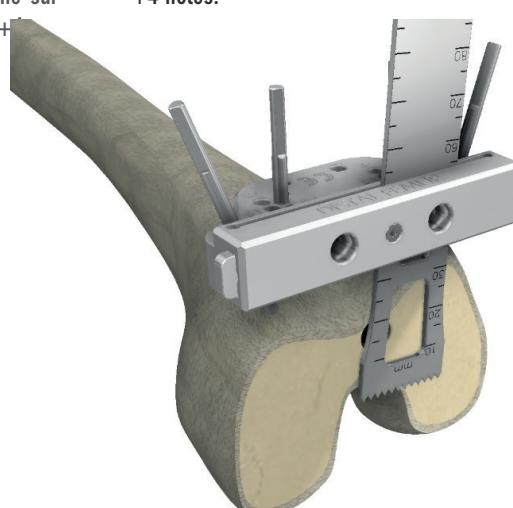
Carry out the distal femoral resection. Check the planarity of the cut using the Angel Wing and if necessary refinish the cut. If oblique pins have been inserted, remove them. Remove the Cutting Block by sliding it outwards. Leave the two parallel Pins in for the next steps.

If necessary, a further resection of 2 mm or 4 mm may be made by repositioning the Cutting Block on the Pins using the +2 or +4 holes.

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Check there is the right gap and correct balancing of the tension of the ligaments in extension using the Spacer Block, starting from the 10 mm size (300110162) and, if required, trying the 12mm (300110163), 14mm (300110164) and 16mm (300110165) sizes.

The Spacer Block must be used from the side marked EXT and is asymmetrical to limit interference with the patellar ligament.

The Alignment Rod (300110161) can be used to verify the correct alignment of the mechanical axis in the distal and in proximal directions.

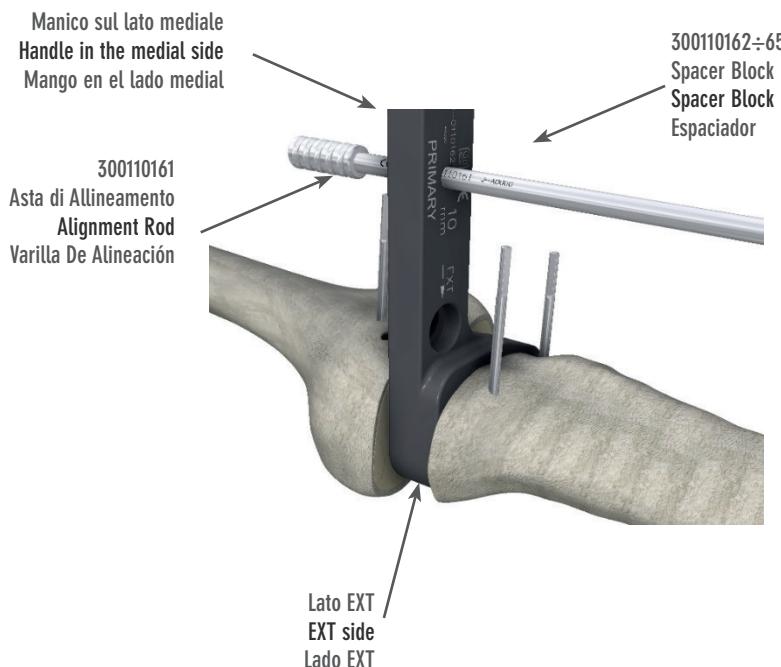
If necessary, the surgeon may decide to perform a further resection of the femur or tibia by repositioning the relative Resection Block on the Pins.

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1

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[3REF FEMUR: MEASUREMENT AND RESECTIONS 4IN1]

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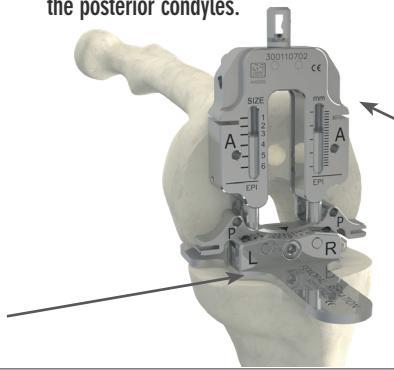
Assemble the Femoral Sizer Extra-Rot (300110702) and the Fem ExtraRotation Guide (300110703). It's possible to pre-select the femoral extra-rotation value (from 0° to 9°). Loosen the locking screw with the Hex Screwdriver For Screws 4,5-6,5mm (340085045), select the desired value with the tip of the arrow, check the L (Left) and R (Right) marks depending on the leg being operated, and then tighten the screw again. With the leg at about 90° flexion, put the assembled components into position and make sure that the Femoral Sizer Extra-Rot is completely in contact with the distal femoral cutting plane and that the adjustable Fem ExtraRotation Guide is in contact with the posterior condyles.

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To check the correct femoral extra-rotation it is possible to insert one or two Guide Wires (300110175) aligning them with the selected reference lines: the Whiteside line and/or the transepicondylar axis. If necessary, release the 3REF Fem ExtraRotation Guide by loosening the locking screw with the Screwdriver, manually rotate the 3REF Fem Sizer to correct alignment, tighten the locking screw to secure the final position of the 3REF Fem ExtraRotation Guide.

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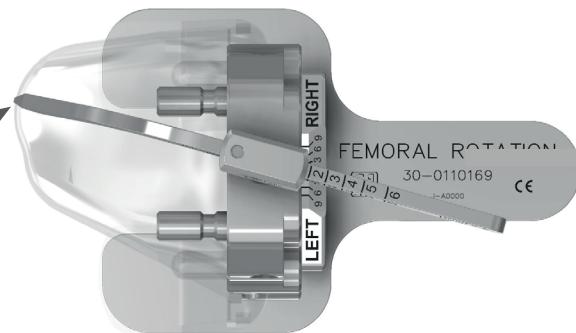
Insert the Femoral Stylus (300110167) in the Alignment Guide as per the FEMUR SIDE indication, which will point in the direction of the femur. Set the Femoral Stylus to the size measured during the preoperative planning, sliding it through its support and positioning the Femoral Stylus tip on the anterior cortex of the femur at the desired run-out point of the anterior resection.

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The selected point represents the anterior reference and should therefore be preferably positioned towards the highest lateral part, in order to avoid the problem of anterior femoral notching.

Read the femoral size through the front slot of the Femur Sizer; if it does not correspond to the stylus setting, reset the stylus and and recheck the size.

If the indicator is midway between two sizes, the surgeon may choose the smaller or the bigger.

In the ANTERIOR reference TOUCH instrumentation procedure, the anterior resection is in the same position for each size of the femoral component.

By choosing the larger size, a larger portion of the bone in the posterior condyles is preserved, the flexion gap is tightened and the risk of excessive ligament tension is decreased, with consequent reduction in the Range of Motion. By choosing the smaller size, more bone in the posterior condyles is resected, the gap in flexion is opened: the Range of Motion will probably be better but there is a risk of more laxity in flexion.

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To check the position of the anterior resection, insert the Angel Wing into the upper, lateral or medial slot of the 3REF Femur Sizer.

Insert two 3,2mm x 80mm Pin (300110144) into the upper holes of the 3REF Femur Sizer indicated with the letter A; rotate the stylus on the medial side and remove the assembly from the femur.

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POSTERIOR REFERENCE

Insert two 3,2mm x 80mm Pin (300110144) into the upper holes of the 3REF Femur Sizer indicated with the letter P.

To check the position of the anterior resection, insert the Angel Wing into the upper, lateral or medial slot of the 3REF Femur Sizer. Place the tip of the Femoral Stylus (300110167) on the anterior cortex towards the highest lateral part in order to avoid the problem of anterior femoral notching.

Read the femoral size through the front slot of the Femur Sizer; if it does not correspond to the stylus setting, reset the stylus and recheck the size. If the indicator is midway between two sizes, the surgeon may choose the smaller or the bigger.

In the POSTERIOR reference TOUCH instrumentation procedure, the posterior resections are the same for each size of the femoral component and, in neutral rotation, equal to the posterior size of the femoral component (9mm).

By choosing the larger size, a larger portion of anterior bone is preserved, the risk of notching is reduced, but, there is the risk of excessive tension in the patellofemoral joint. By choosing the smaller size, a smallest portion of anterior bone is resected; the possibility of excessive tension in the patellofemoral joint is reduced, but there is a risk of creating a femoral notching. In order to limit this risk, the anterior femoral resection develops a 4° inclination.

Rotate the stylus on the medial side and remove the assembly from the femur.

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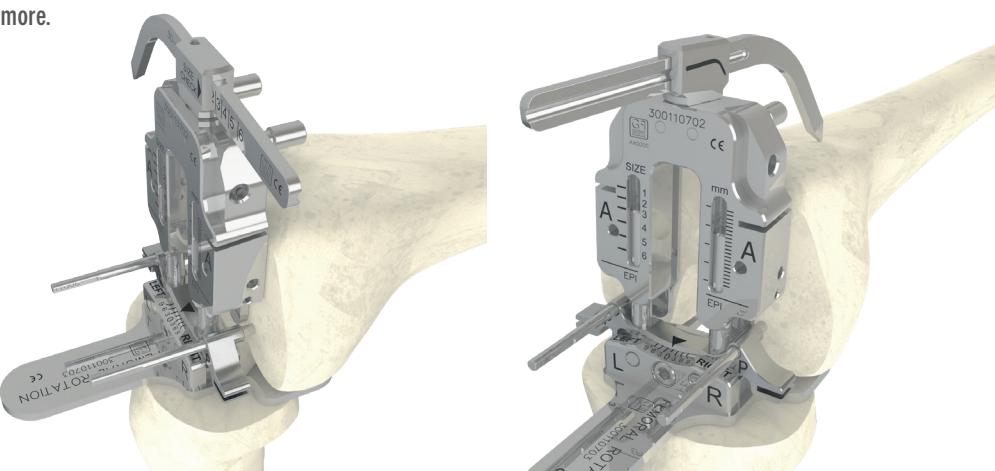
FEMORAL RESECTIONS 3REF 4IN1

The 3REF Resection Block 4IN1 (300110176÷81) has an outside measurement in the ML plane equal to that of the relative femoral component and it also shows the marks relating to the measurement in ML also for the Skinny version (sizes 2-, 3-, 4-).

Check for correspondence between the size chosen in AP and its measurement in ML.

Should it be necessary to scale the femoral component up or down, see the previous recommendation based on the type of reference chosen.

Should the size in AP be adequate but the measurement in the ML plane be excessive, for sizes 2, 3 and 4 it's possible, in the final phase of the implant, to use the Skinny version (sizes 2-, 3-, 4-).

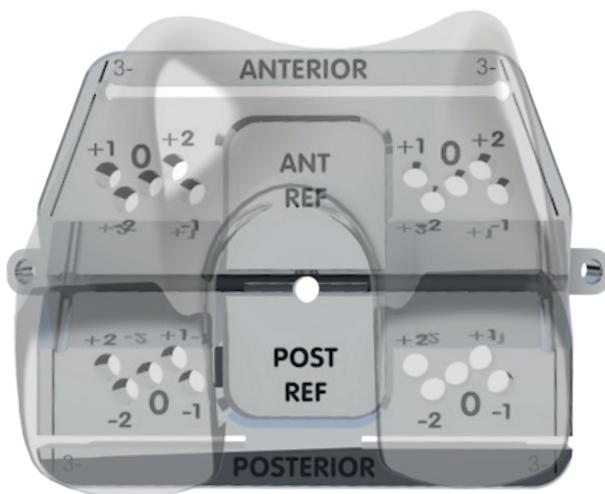
For example, Skinny size 3- has an outside measurement in AP equal to size 3 but an outside measurement in ML equal to size 2. To implant this, the surgeon should use the 4in1 Resection Block 4IN1 and the size 3 Trial Femoral.

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Check the ML dimension, especially anterior.

- 3REF Resection Block 4IN1 and articular geometry: same as the reference size;
- Measurement in AP: equal to the reference size;
- Measurement in ML: equal to that of the next size down, equivalent to a symmetrical reduction of 2 mm on both sides.

The position of the two Pins in the ML plane, and therefore the position of the 3REF Resection Block 4IN1, does not constrain the final positioning of the definitive component in that plane.

Position the 3REF Resection Block 4IN1 corresponding to the chosen measurement on the Pins, using the 0 holes, consistently with the reference previously chosen and reported on the block (ANT REF or POST REF).

Slide it until it fully adheres to the distal resection plane of the femur.

Before proceeding with femoral resections, perform the following checks.

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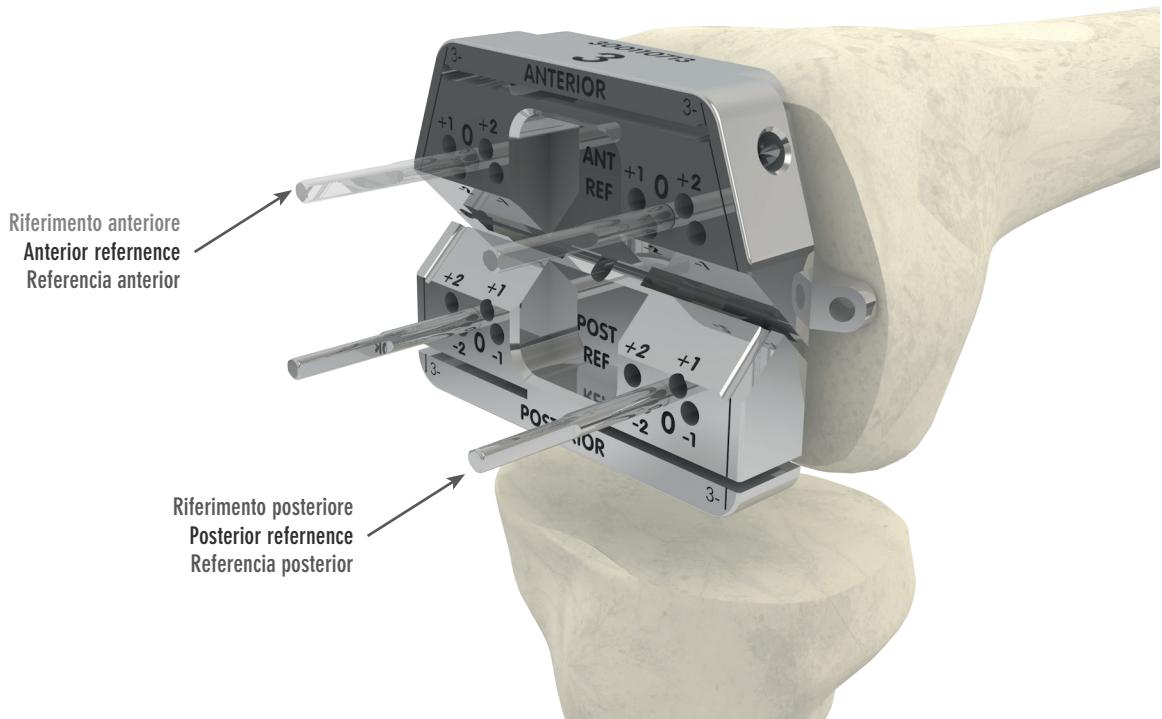
ANTERIOR RESECTION CHECK

Check the position of the anterior resection with the help of the Angel Wing.

In order to avoid the problem of anterior femoral notching, the exit angle of the anterior resection should be equal to 4°.

If necessary, correct the level of the anterior resection by translating it posteriorly or anteriorly by 1mm or 2mm, by repositioning the 3REF Resection Block 4IN1 in the holes "+1, +2 or -1, -2", always in correspondence with the reference previously chosen and shown on the block (ANT REF or POST REF).

Attention: the displacement of the level of the anterior resection affects the width of the flexion gap (see next paragraph).

Posterior reference: In this case it is possible to vary the level of the anterior resection also by changing the size of the femoral component. For example, in case of danger of notching, it is possible to evaluate a higher measure that shifts the resection level towards the front. The difference in AP between two different femoral sizes is 4mm, while between size 5 and 6 it is 5mm. Oversizing or undersizing of the femoral component does not alter the posterior femoral reference.

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FLEXION GAP

Check the gap and the ligament balancing in flexion using the Spacer Block of the thickness determined for the extension gap. Inserted from the side marked FLEX, it must be flat up against the tibial resection and the lower part of the 3REF Resection Block 4IN1. The Spacer Block and Resection Block reproduce the total thickness of the prosthesis in flexion. If necessary, the width of the flexion gap can be changed: moving the Resection Block posteriorly or anteriorly into the holes marked +1, +2 or -1, -2, always in correspondence with the reference previously chosen and shown on the block (ANT REF or POST REF). Attention: moving the Resection Block to vary the width of the flexion gap affects the level of the anterior resection (see previous paragraph).

Anterior reference: In this case it is possible to vary the width of the flexion gap also by changing the size of the femoral component. For example, in case of danger of a tight flexion gap, it is possible to evaluate a lower measure that moves the resection level of the posterior condyles towards the anterior. The difference in AP between two different femoral sizes is 4mm, while between size 5 and 6 it is 5mm. Over or under sizing of the femoral component does not alter the anterior femoral reference.

If the contact between the Resection Block and the Spacer Block is not uniform, a trapezoidal bending gap could be created instead of rectangular; in this case, consider a ligament correction and/or a different femoral extra-rotation setting.

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
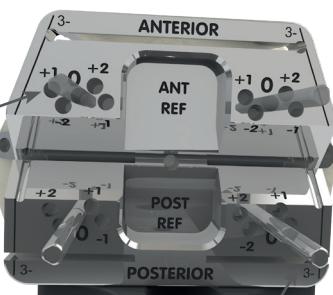
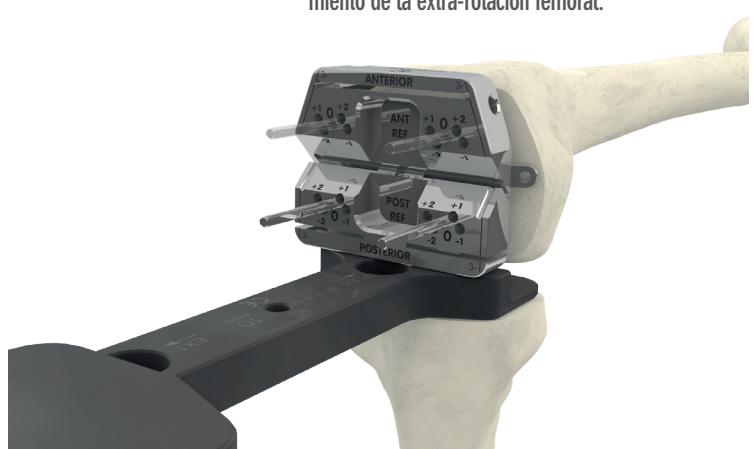
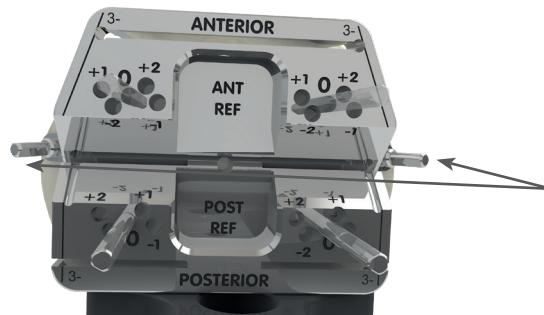
Fix the 4in1 Femoral Cutting Block in the final position by putting two Threaded Pin Ø3.2mm- Ø4,2mm (300110379) or two Pin With Stop 3.2mm x 40mm (300110198) in the oblique holes.

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Make the anterior and posterior resections; remove the parallel Pins and proceed with the oblique anterior and posterior cuts. For greater stability it's possible to put a Pin in the central hole of the Cutting Block and/or apply the Universal Handle (300110279) and Universal Handle/Pin Extractor (300110281). Remove the Pins and the 4in1 Femoral Cutting Block.

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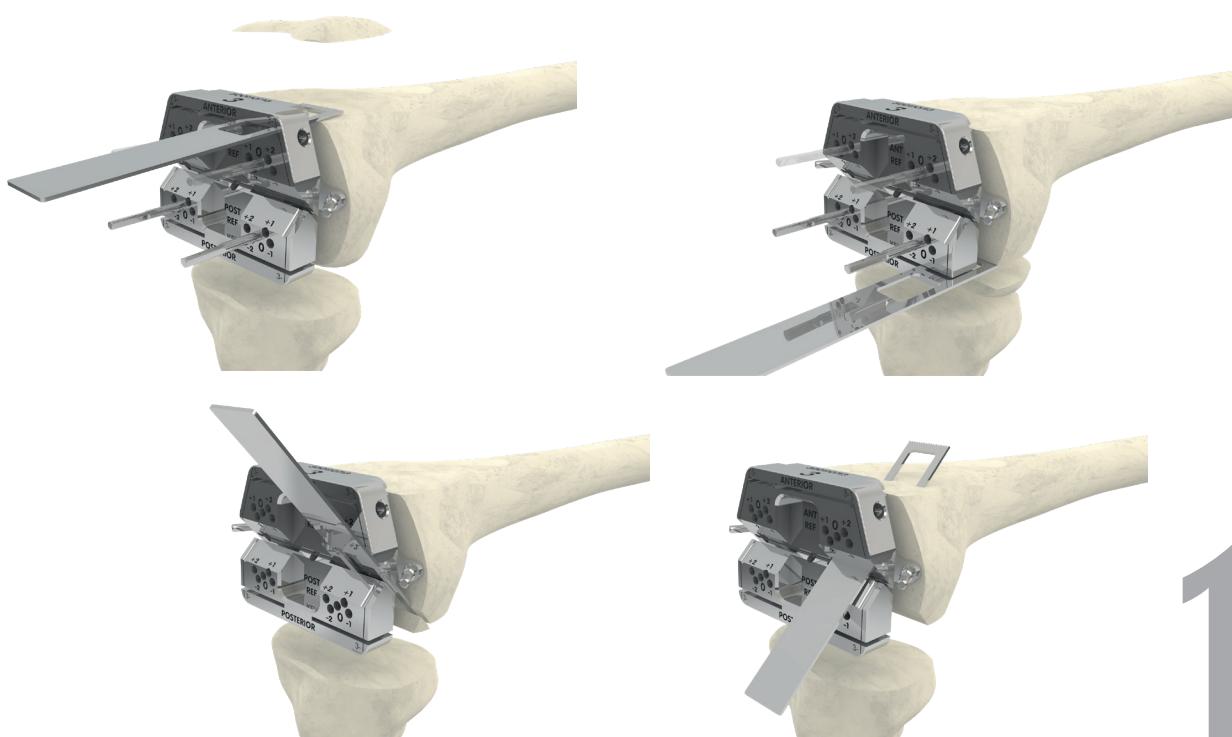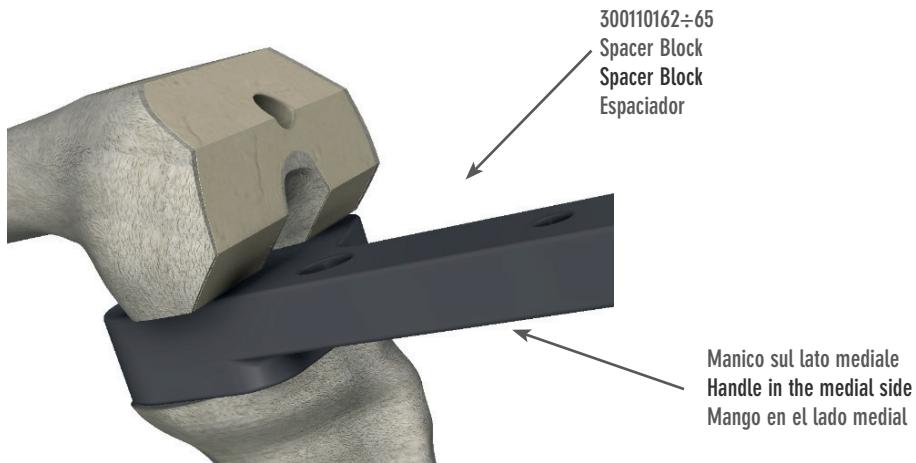
Check there is the right gap and correct balancing of the tension of the ligaments in flexion using the Spacer Block defined for the extension gap.
The Spacer Block must be used from the side marked EXT and is asymmetrical to limit interference with the patellar ligament.

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Position the femoral box guide K-MOD PS Fem Box Guide (300111101÷106) for the chosen size on the anterior, distal and anterior oblique femoral planes. Place it in a central position, or slightly lateral, on the ML plane. The medial side of the guide goes on the medial side of the definitive component. In the case of Skinny femoral components (2-, 3-, 4-), the size is indicated on them.

Make sure there is perfect adherence between the osseous beds and the metal surfaces of the guide.

Fixate the guide using three pins, two of which positioned in the holes on the distal and anterior oblique surfaces (medial and lateral) and one on the anterior surface. Use the Threaded pin Ø3,2mm- Ø4,2mm (300110379) or Pin With Stop 3,2mm x 40mm (300110198) or Headed Pins (300110197).

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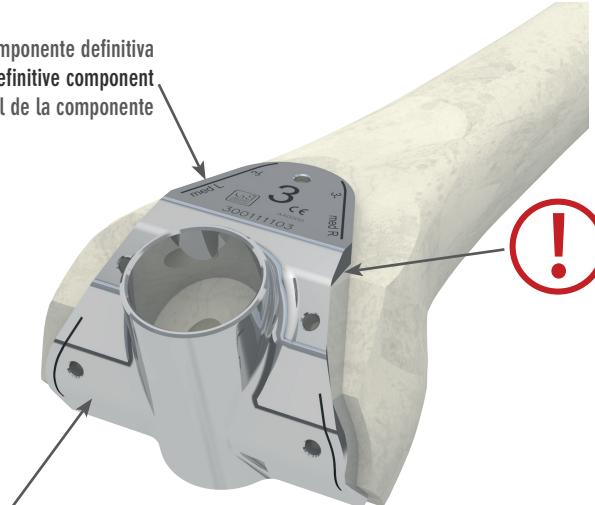
WARNING: The lateral side of the PS Box Guide does NOT correspond to the lateral side of the final component!

DO NOT lateralize the femoral component using the lateral wall of the PS Box Guide as a reference

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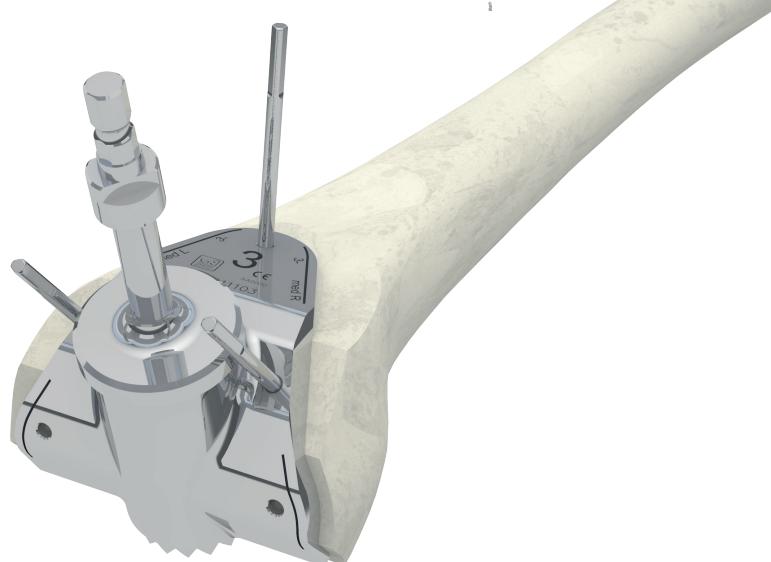
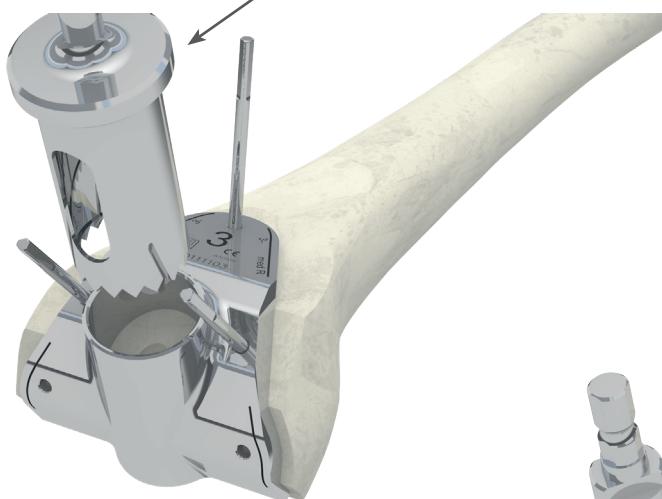
Select the reamer K-MOD PS Drill Box Fem (1-2, 3-4, 5-6: 300111121÷123) for the chosen femur size. Position it axially in the cylinder in the PS Fem Box Guide and produce the PS femoral box by advancing the reamer up to the stop.

Remove the reamer and PS Fem Box Guide by extracting the Pins and. If Headed Pins are being used, use the Universal Handle/ Pin Extractor to remove them.

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Assemble the Femoral Trial (300110223÷34; 300111201÷26) of the chosen size and for the side of the leg operated (L: Left; R: Right) on the Femoral Positioner (300110235).

Should it be necessary to use a Skinny component with a reduced measurement on the ML plane (sizes 2-, 3-, 4-), the Femoral Trial to use will in any case be the reference size (2, 3, 4).

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Position the Femoral Trial on the femur; the recommended position on the ML plane is central or slightly lateral to facilitate patellar slide; in case of PS femoral component, refer to the cylindrical slot made with the Drill Box. Avoid protrusion of the component with respect to the bone.

The ML measurement of the Skinny femoral components is symmetrically reduced by around 2 mm on both sides compared to the reference size Trial.

Depending on the inclination of the anterior resection, impact making sure the handle of the Femoral Positioner is turned upwards, so as to maintain congruence between the resections and the Trial's internal surfaces.

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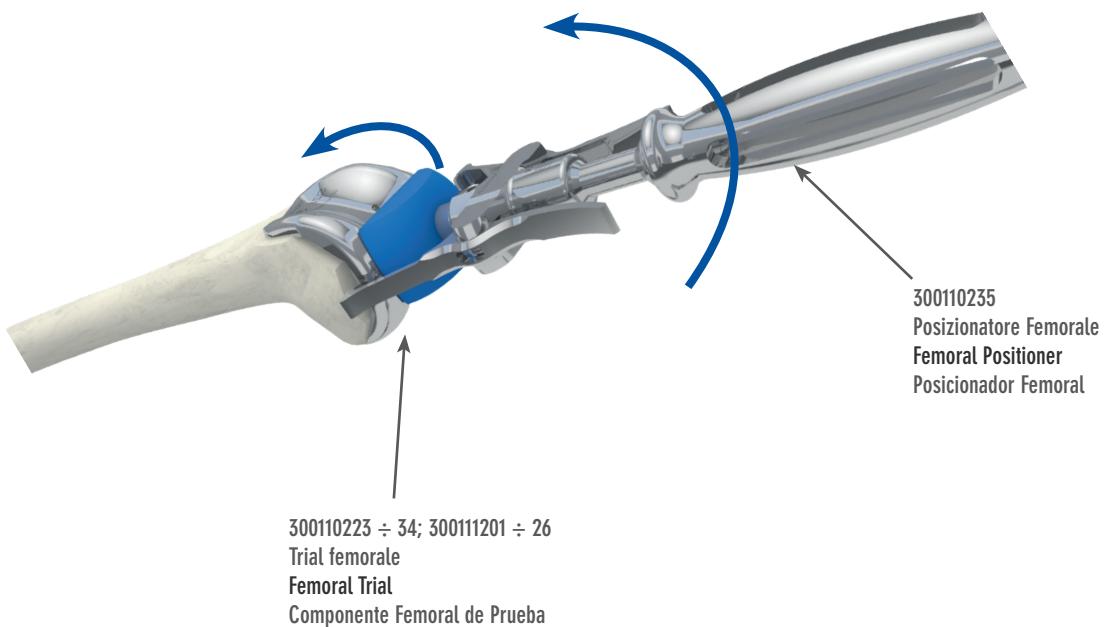
Release the Positioner and complete the positioning with the Femoral Impactor (300110259).

Make sure the Trial is perfectly adherent to the resections and check for any medial-lateral protrusion.

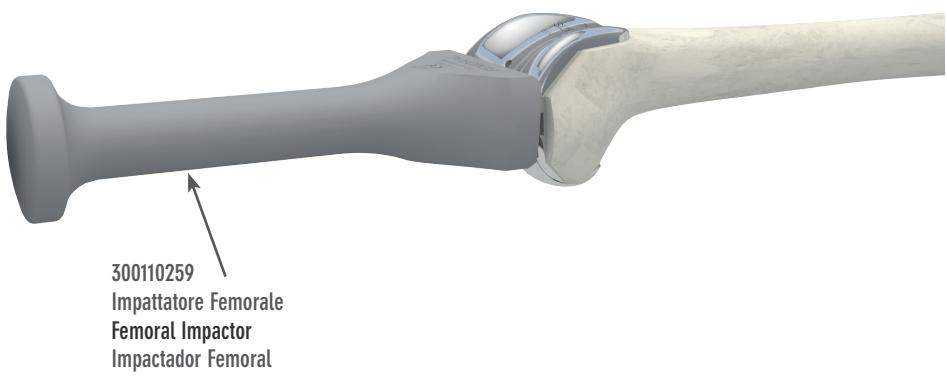
NB.: it is also possible to insert the Femoral Trial after positioning the tibia trial components. In this case, position it in deep flexion and impact with the leg at around 90° flexion.

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
Using the Tibial Baseplate (300110190÷95), select the tibial size that provides the desired tibial coverage.

The Baseplate can be assembled onto Tibial Baseplate Handle (300110682) by means of the two anterior holes; on rotating the central nut, the two pins of the Handle close (LOCK) or open (RELEASE) to hold or release the Baseplate.

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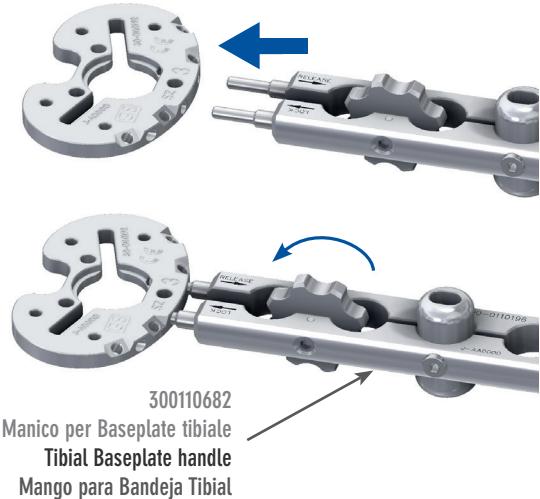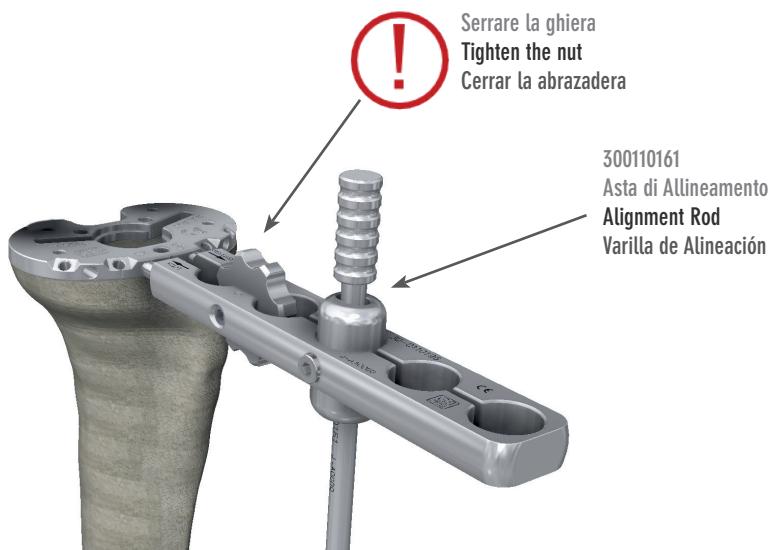
Correct positioning of the Baseplate and alignment with the tibial mechanical axis can be checked by inserting the Alignment Rod (300110161) in the Handle. Tighten the nut well to prevent the Rod from shifting in the varus/valgus plane.

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If desired, it is possible to fix the positioning of the Baseplate with Headed Pins (300110197) in the anterior oblique holes. These pins in any case make it possible to position the trial insert and correctly perform the trial reduction. They can be removed with the Universal Handle/Pin Extractor (300110281).

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300110401÷36; 300111401 ÷436

Trial Tibiale Mobile
Trial Mobile Insert

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300110395

Perno Inserto Prova Mobile
Mobile Insert Trial Peg
Pivot Inserto Prueba Móvil

[K-MOD PS]

Release the Handle from the Baseplate and position the Trial Insert on the Baseplate by choosing one of the following options

- **FIXED INSERT:** posterior cruciate ligament retaining CR (300110441÷76), Ultra Congruent UC (300110481÷516), Dynamic Congruence DC (300110521÷96) depending on the side of the leg being operated (L: Left; R: Right) or Posterior Stabilized PS (300111301÷336). The Trial is applied by fitting the lower part into the special slot in the Baseplate.
- **MOBILE INSERT:** UltraCongruent UC (300110401÷436) or Posterior Stabilized PS (300111301÷336); put the Mobile Trial Insert Peg (300110395) in the Baseplate. The Trial hooks onto the Peg by means of a special hole in the bottom, thus remaining free to rotate.

NB.: If desired, it is possible to leave the Handle in during the trial.

The size of the tibial insert corresponds to that of the femoral component, even if Skinny femoral components are subsequently used.

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Assess the stability of the joint in extension and flexion. The extension check should be performed with the knee flexed a few degrees to relax the posterior capsule. However, the knee should be able to extend fully. The flexion check should be performed with the knee flexed to 90°.

Optimum stability is achieved when the medial and lateral opening is similar to that of a normal knee under valgus and varus stress. An adjustment of ligament balance may be needed if there is differential ligament tightness between varus and valgus in flexion or extension.

After achieving the correct rotational position of the tibial plate and making sure that does not protrude beyond the tibial profile, the surgeon can mark the tibia by the frontal marks. This will be the landmark for repositioning the Baseplate and preparing the bed for the tibial stem.

The Baseplate can also be fixated using the pins in the anterior oblique holes.

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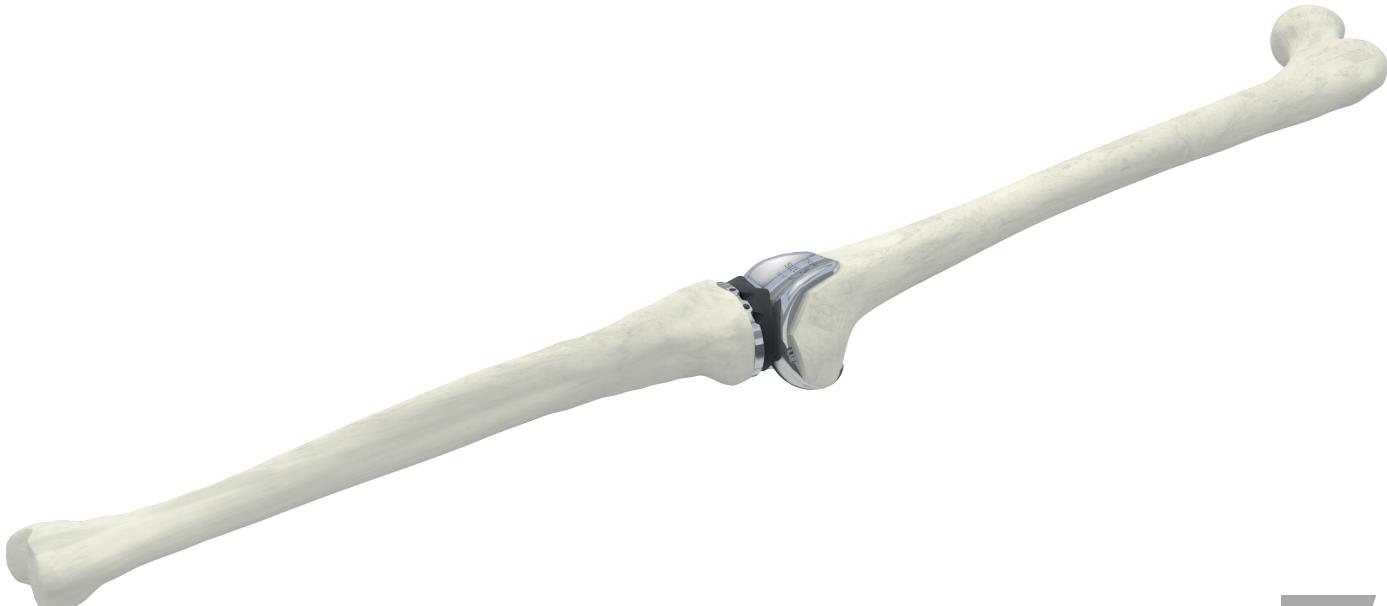
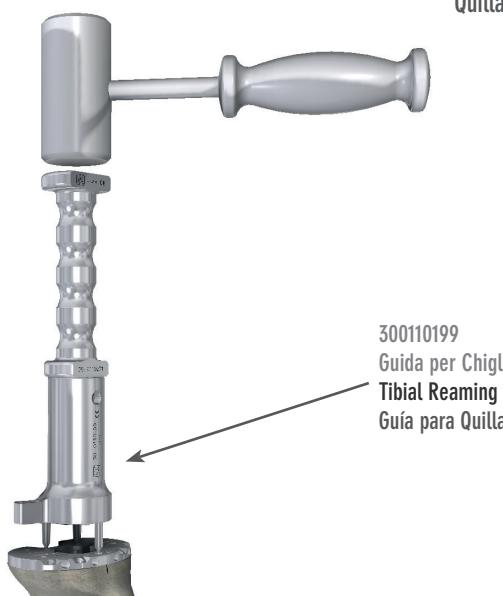
Remove the Trial insert and, in the case of a mobile insert, also the Mobile Trial Insert Peg (300110395). Insert the Tibial Reaming Broaching Keel Guide (300110199) in the Baseplate by impacting the three spikes into the proximal tibia through the holes in the Baseplate. For impacting, it's possible to insert the Tibial Broach Handle (300110271) in the Keel Guide.

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
Prepare the conical slot in the bone using the Conical Tibial Reamer (300110200) and inserting it into the Guide as far as it will go.

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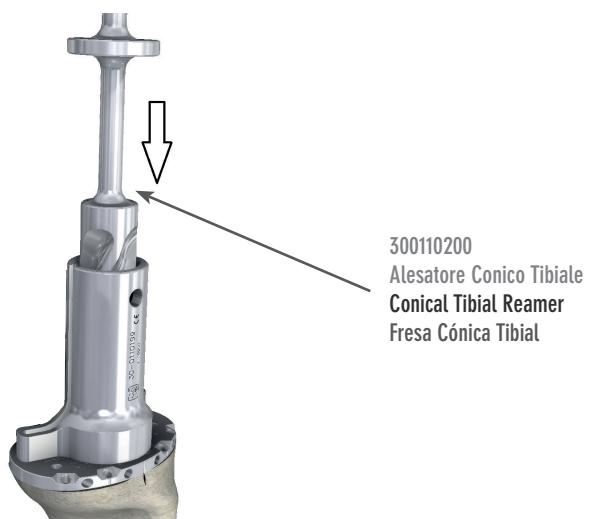To prepare the slot for the keel, select the Keel Broach (Sizes 1-2: 300110254, Sizes 3-4: 300110255, Sizes 5-6: 300110256) corresponding to the size of the chosen tibial Baseplate. Assemble the Broach on the Broach Handle (300110271) and impact it through the Guide as far as it will go.

To limit the risk of fracturing of the tibial tray in the case of particularly sclerotic bone, it is best to prepare the slots for the fins using a small osteotome.

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Valid procedure for femoral component NOT posterior stabilized PS

With the Femoral Trial properly positioned, make the housing for the two femoral pegs by inserting the Femoral Peg Drill (300110246) in the two holes on the Trial and proceed as far as the stop.

NB.: if Skinny femoral components (2-, 3-, 4-) are subsequently to be used, use the reference size Trial (2, 3, 4). When positioning it, bear in mind that the outside measurement of the Skinny component in the ML plane will be symmetrically smaller by around 2mm on both sides.

It's advisable to check for the presence of osteophytes or protrusions on the posterior condyles with respect to the profile of the femoral component; such features could limit flexion and should be removed.

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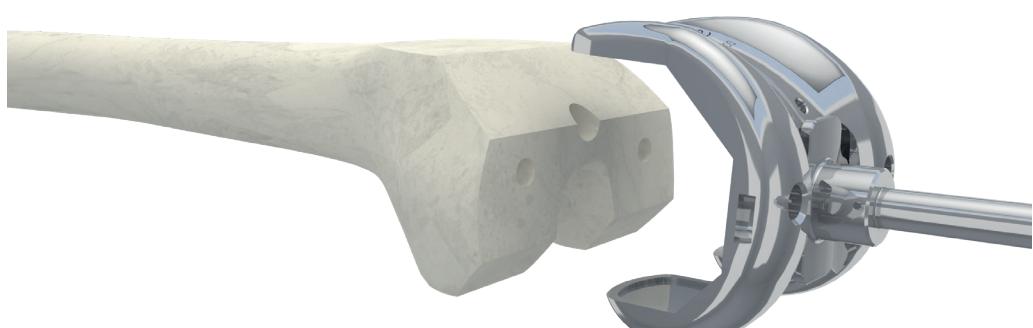
Remove the Trial Femoral using the Slide-Hammer (300110277), which has a plate on one end for coupling with the Trial itself. The narrower end of the plate is recommended for sizes 1 and 2, while the wider end is for sizes 3 to 6.

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Choose the definitive components with great care. The colour code on the packaging of the femoral components and inserts refers to the colour of the Trial Insert defined during the trial reduction.

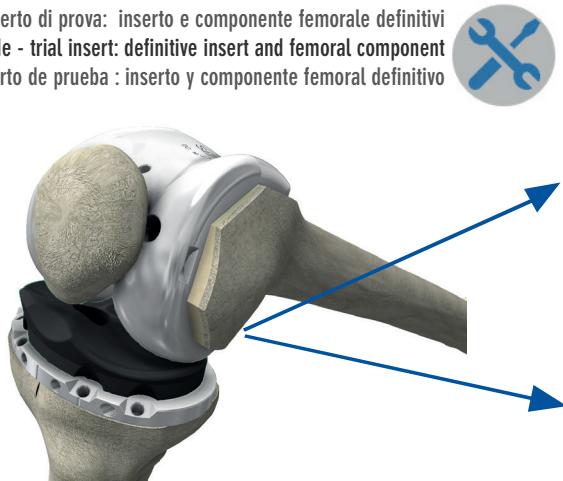
The size of the polyethylene insert corresponds to that of the femoral component, even when Skinny femoral components are used. The package of the fixed tibial component includes the locking screw of the polyethylene insert.

For both fixed and mobile tibial components, it is advisable to always check the effective locking of the tibial stem closure cap using the Hex Screwdriver For Screws 4.5-6.5mm (340085045).

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Inserto Mobile
Mobile insert
Inserto Movil

Inserto Fisso
Fixed insert
Inserto Fijo

When using cemented components, follow the reference cementing technique for the operating surgeon. Then carefully apply the cement.

Thoroughly clean the surfaces that will house the implants, eg. with pulse lavage, and position the knee in deep flexion.

Luxate the tibia anteriorly and insert the tibial component using as reference points the marks already made and the preparation carried out for the keel.

Complete the positioning using the Tibial Tray Impactor (300110258) and make sure there is perfect adherence between the component and the cut.

When using cemented components, carefully remove all excess cement, especially in the posterior zone.

Thoroughly clean the upper surface before proceeding with the polyethylene implant.

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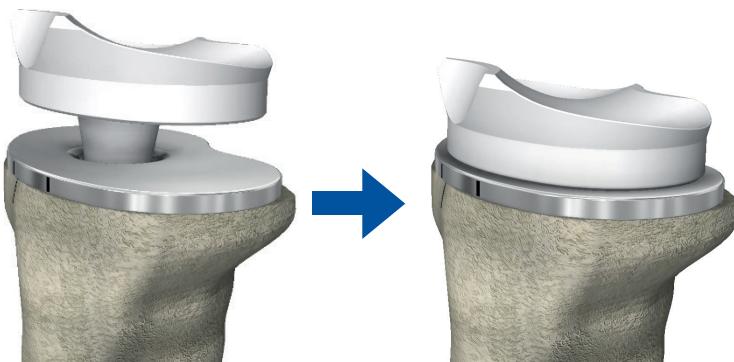
Inserto Mobile
Mobile insert
Inserto Movil

MOBILE INSERT

Insert the rotating implant in the tibial tray. The ANT indication must be on the front side of the tibia

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Inserto Fisso
Fixed insert
Inserto Fijo

FIXED INSERT

Introduce the insert into the tibial tray by engaging the posterior side first, and keeping it inclined. The groove on the posterior edge of the insert must fit into the posterior profile of the tibial tray.

Now bring the anterior part into contact as well and apply force obliquely on the front edge of the insert so it can hook into the anterior profile of the tibial tray.

Using the Screwdriver (340085045), fully tighten the safety screw for micro-motion presents in the tibial component pack.

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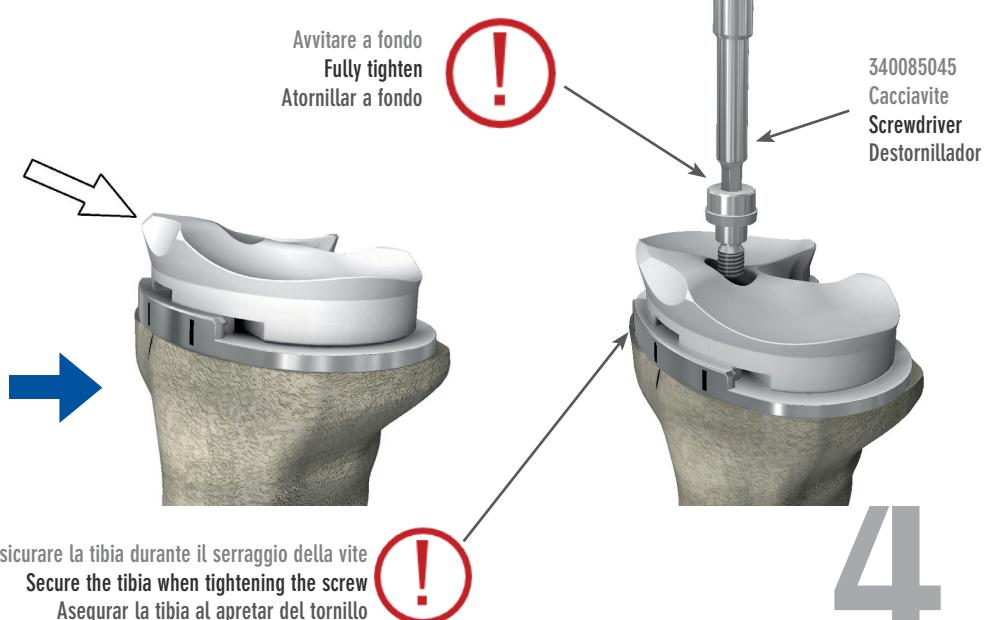
FIXED INSERT: SAFETY LOCKING SCREW

It's possible to tighten the safety screw for micro-movement AFTER implanting the femoral component. This makes it possible to check that the chosen thickness is adequate. Make sure that the hole for the screw is completely unobstructed and clean before starting to insert the screw. Should it be necessary to change the insert AFTER anchoring it with the safety screw, unscrew it as far as the exit from the tibial plateau. Warning: the safety screw will in any case remain housed in the hole of the polyethylene insert! Remove the insert by leveraging upwards on the anterior wall and replacing it with a new one. Secure it using the screw in the single packet K-MOD Lock Screw x Fixed Insert M5 (120700701), which can be used with any size because micro-movement safety screws for K-MOD fixed tibial components are all the same.

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Assemble the femoral component on the Femoral Positioner (300110235).

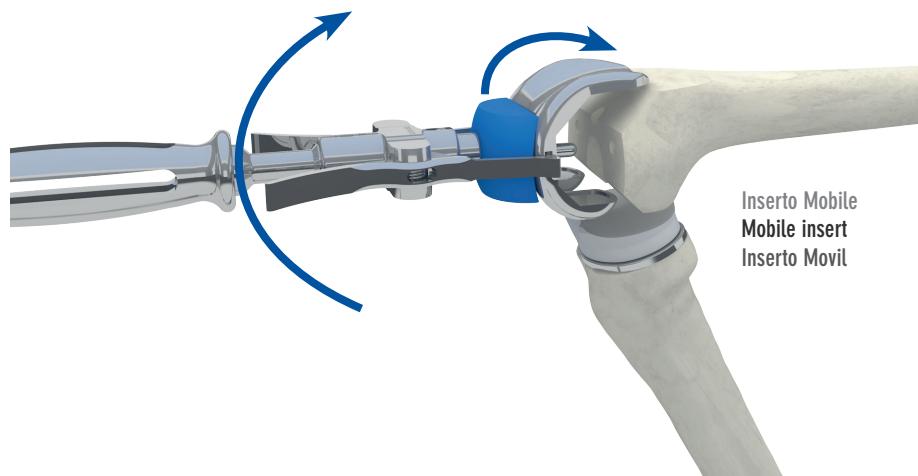
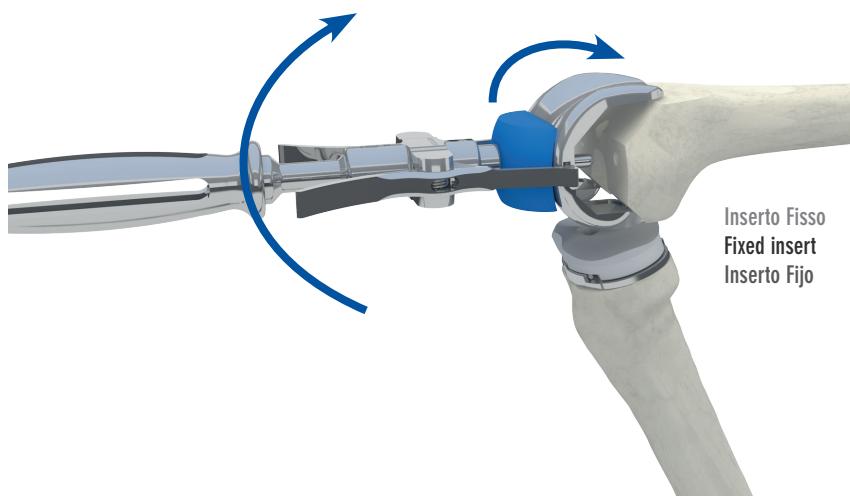
To hold or release the component, rotate the handle and close or open the hooks.

Put the knee in deep flexion. Position the femoral component on the femur starting from the posterior edge in contact with the posterior resections and making sure that the two pegs fit into the relative holes. Impact with the handle of the Femoral Positioner turned upwards, thus rotating the component from posterior to anterior, to obtain congruence between the resections and its internal surfaces.

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Inserto Mobile
Mobile insert
Inserto Movil

7

Now adjust the joint to 90° flexion. Disconnect the Positioner and complete positioning with the Femoral Impactor (300110259). Check there is perfect adherence between components and cuts. When using cemented components, carefully remove all the excess cement.

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Settle the components by fully extending the leg. When using cemented components, leave the leg in extension during the cement setting phase. Check and then proceed to remove any excess cement.

Assess knee movement and stability within the Range of Motion. Thoroughly irrigate and clean the joint of all residues. Proceed with suturing and closure in the surgeon's preferred manner.

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[OPTION: INTRAMEDULLARY TIBIAL RESECTION]

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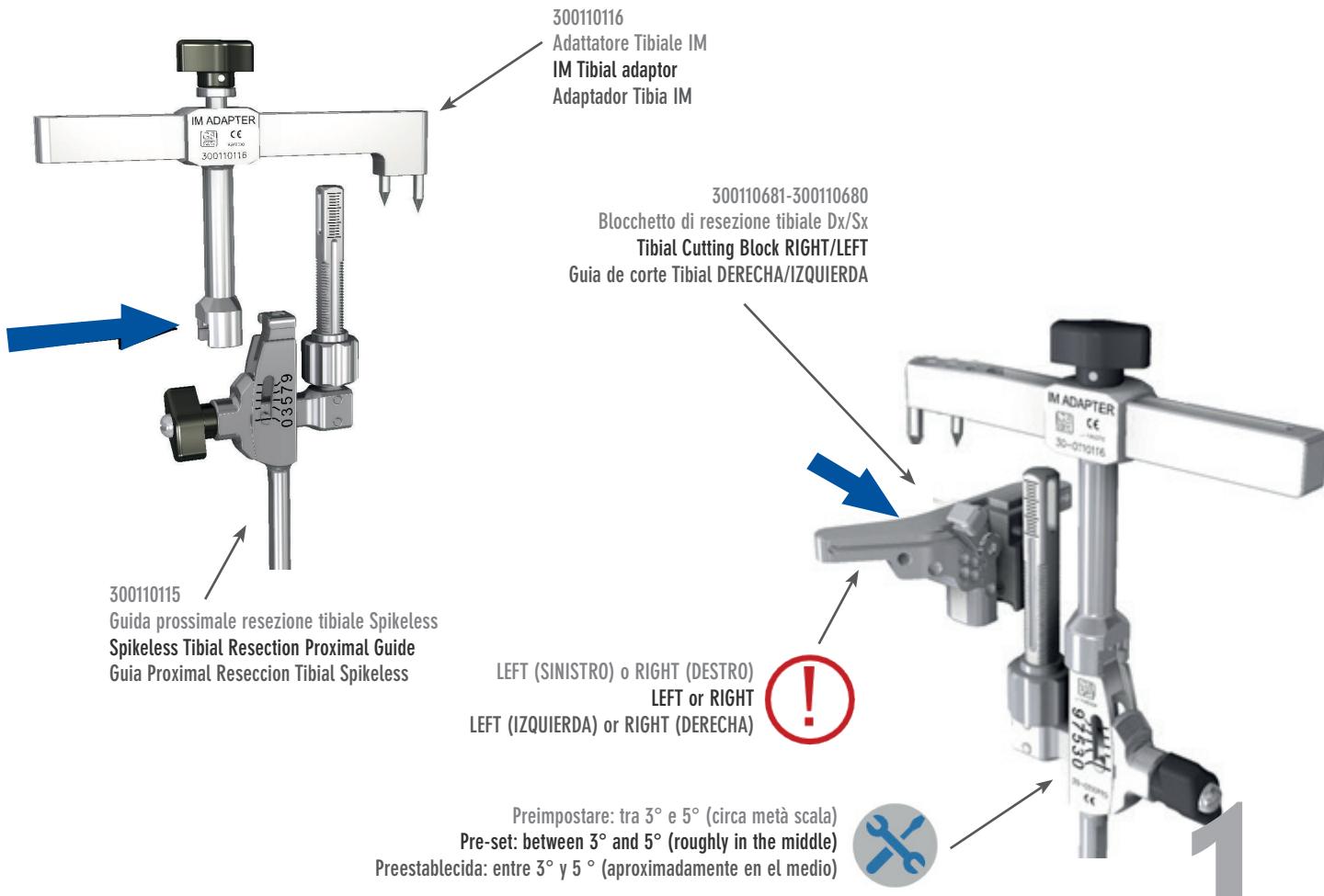
The intra-medullary tibial resection guide consists of three parts: the Spikeless Tibial Resection Proximal Guide (300110115), the appropriate Tibial Cutting Block (RIGHT 300110681, LEFT 300110680) and the IM Tibial Adaptor (300110116).

The guide must be assembled before application. Raise the top knob on the IM Tibial Adaptor to lift the connection pin, mount the Spikeless Tibial Resection Proximal Guide, tighten the knob on the top.
Make sure you've chosen the appropriate Tibial Cutting Block (LEFT or RIGHT). Pre-select the desired tibial slope (range: 0°-9°) on the Spikeless Tibial Resection Proximal Guide.

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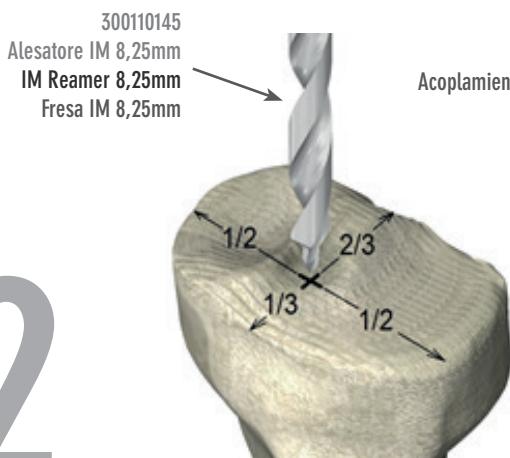
Use the IM Reamer 8,25mm (300110145) to make an intramedullary hole in the tibial channel. The hole must be roughly in the centre of the ML plane and in 1/3 2/3 ratios in AP direction.


Insert the appropriate IM Rod (LONG 300110150, SHORT 300110376) in the hole of the tibial channel with the aid of the T-Handle For IM Rod (300110151). The IM Rod has longitudinal grooves that limit pressurization of the channel and the risk of embolisms.

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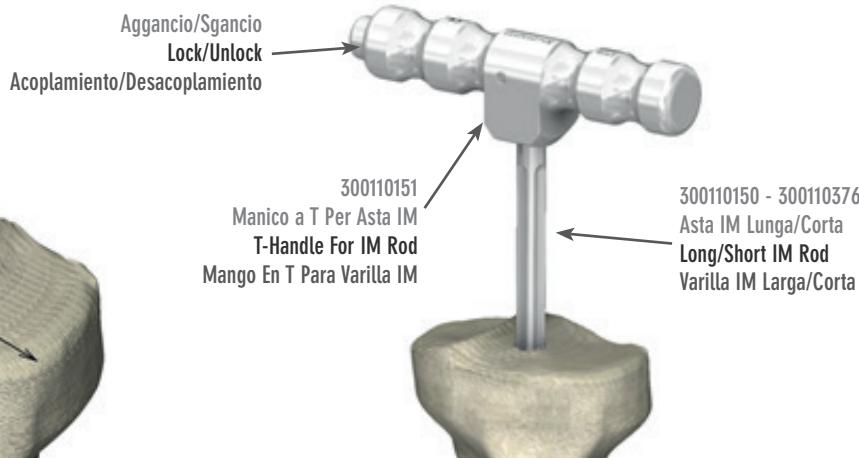
Adjust the height of the Tibial Cutting Block to roughly the level of the desired cut. Align the guide on the proximal tibia with the third medial of the tibial tuberosity. Lock it in position by sinking the two spikes into the tibial bone using a hammer.

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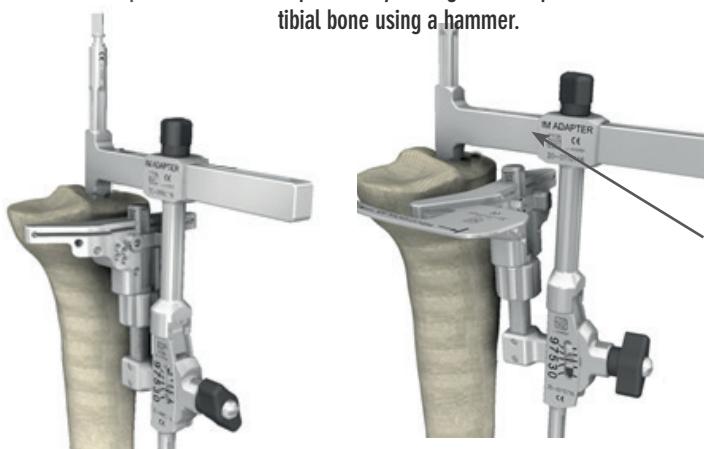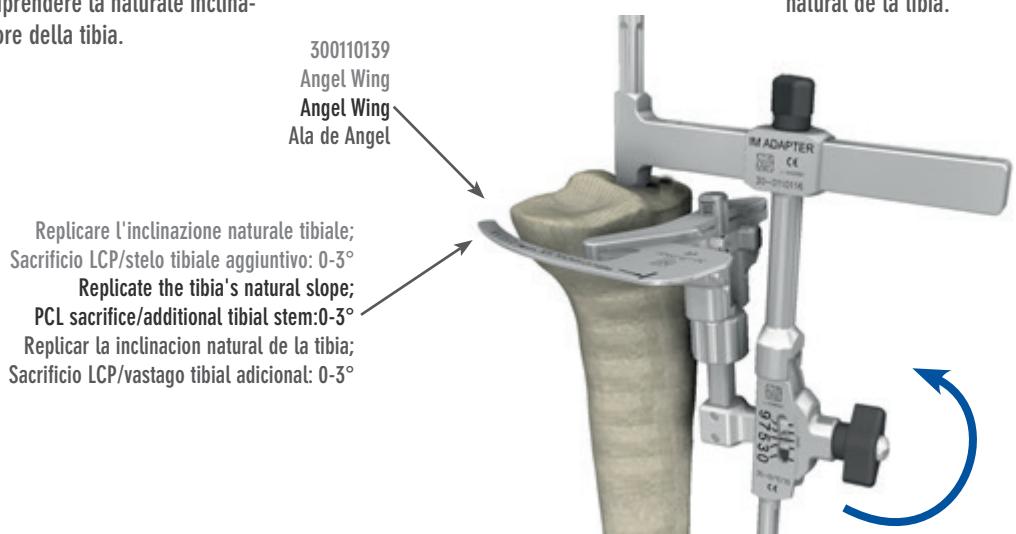
The posterior slope can be adjusted by using the Angel Wing (300110139) and operating the knob on the Spikeless Tibial Resection Proximal Guide.

In general, proximal tibial resection should follow the tibia's natural posterior slope.

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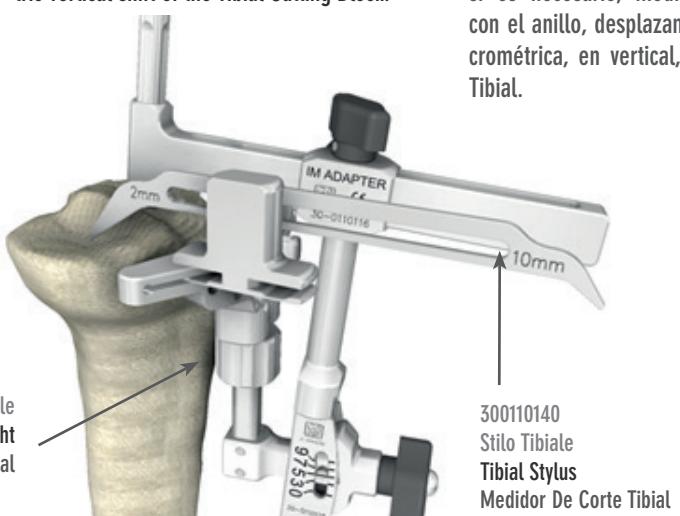
Position the Tibial Stylus (300110140) in the slot of the Cutting Block. The surgeon may choose to use as a cutting reference the lowest point on the most worn side of the tibial plateau and carry out a minimum resection (2mm tip for 2mm resection) or the highest point on the least worn side (10mm tip for 10mm resection).

Check that the cut is adequate and if necessary modify the size of the resection by operating the ring and making a micrometric vertical shift of the Tibial Cutting Block.

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Release the knob, offer up the Cutting Block to the tibial bone. Lock the knob and anchor the Cutting Block with two Pin 3,2mm x 80mm (300110144) in the parallel O rows of holes.

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Remove the guide: raise the Cutting Block trigger and slide the ring downwards. Release the knob of the IM Tibial Adaptor and move it away from the tibia. Extract the assembly by sliding it along the IM Rod.

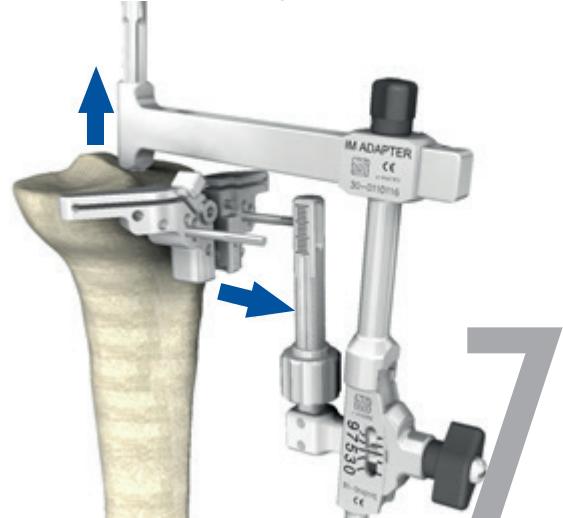
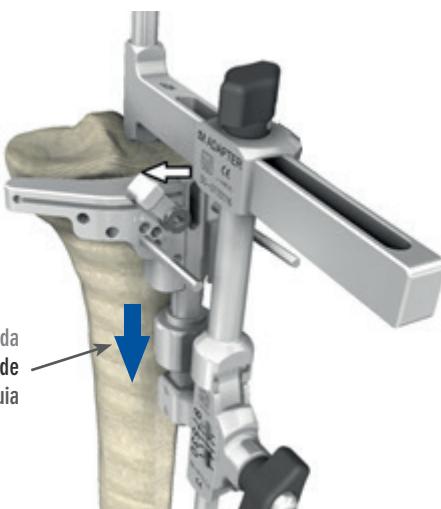
NB: should the opening of the Cutting Block trigger be obstructed by the top arm of the IM Adapter, slide the ring downwards and lift the entire assembly till the trigger can be opened.

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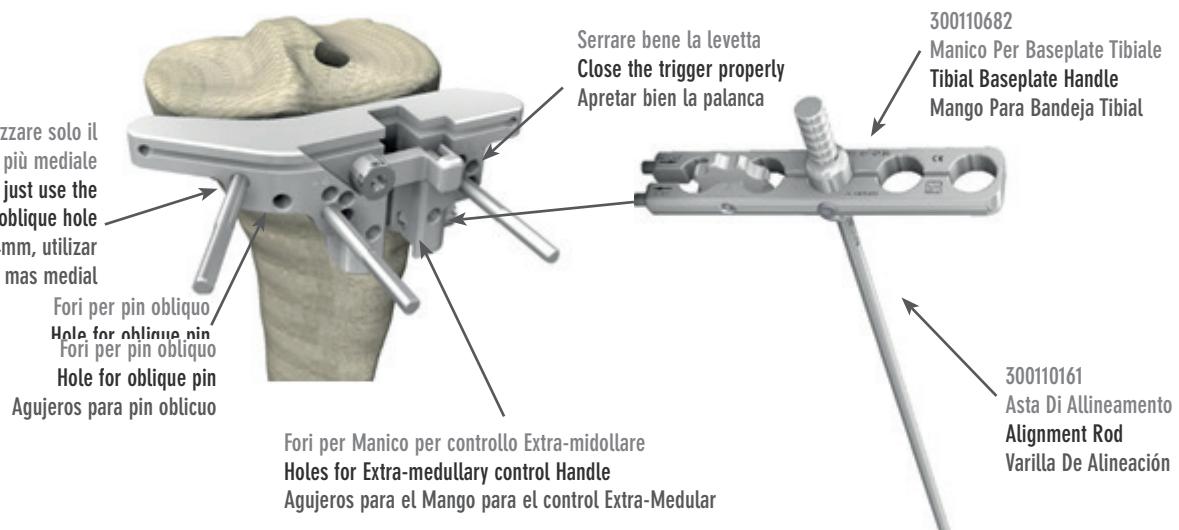
In the case of a +4mm RECUT, just use the more medial oblique hole

En caso de RECUT a +4mm, utilizar solamente el agujero oblicuo mas medial

Fori per pin obliqui
Hole for oblique pin
Fori per pin obliqui
Hole for oblique pin
Agujeros para pin oblicuo

The Cutting Block may be made more stable by inserting a third pin in the oblique holes on the medial side of the Cutting Block, using either the Pin 3,2mm x 80mm (300110144), the Pin with stop 3,2mm x 40mm (300110198) or the Threaded pin Ø3.2mm-Ø4,2mm (300110379). Make sure that the Cutting Block trigger is properly closed. It's possible to check alignment with the mechanical axis by inserting the Tibial Baseplate Handle (300110682) in the appropriate holes on the Cutting Block and the Alignment Rod (300110161) in the Handle. Make sure the Rod is locked and cannot swing.

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Perform the resection of the proximal tibia. Verify the planarity of the cut by using the Angel Wing and if necessary refinish the cut. If an oblique pin has been inserted, remove it. Remove the Cutting Block by sliding it outwards. Leave the two parallel pins in for the next steps.

If necessary, a further 2mm or 4mm resection can be made by remounting the Cutting Block on the pins in the +2 or +4 holes.

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The Tibial Resection Guide assembly consists of four parts: the Proximal Guide (300110114), the appropriate Cutting Block (RIGHT 300110681; LEFT 300110680), the Tibial Distal Guide (300110122) and the Ankle Clamp (300110133). The assembly must be done first. The alignment method used is that the extra-medullary.

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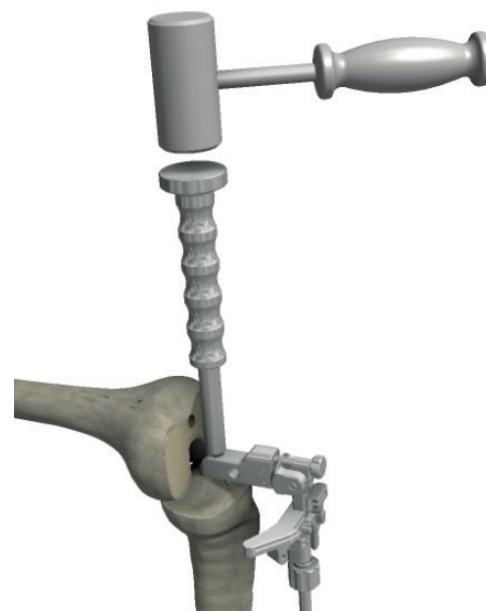
Place the Tibial Guide assembly on the tibia. Make sure that the correct (LEFT or RIGHT) Tibial Cutting Block is chosen and that the Alignment Guide is correctly set distally.

Impact the spikes of the Proximal Guide into the tibia using the Conical Impactor (300110322).

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
Remove the locking shaft from the Proximal Guide. The proximal joint of the guide is free to rotate in two planes: this feature makes it possible to set the correct adjustments of the posterior slope and the Varus-Valgus angles without detaching the spikes from the tibial plateau. In this way, it is possible to mimic the natural posterior slope using the "Angel Wing" (300110139). In general, the natural inclination of the tibial base should be reproduced by the proximal tibial resection.

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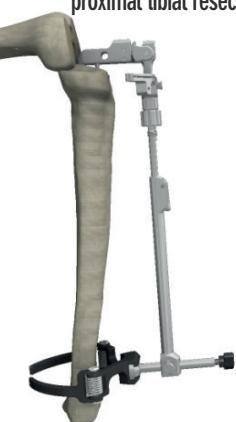The Tibial Stylus (300110140) is placed into the slot of the Cutting Block and the surgeon can choose the less affected or the most affected side of the tibial plateau to use as reference for the cut.

If the surgeon chooses the least affected side, the 10 mm tip of the Stylus should be used, which will produce a 10 mm resection. If the surgeon chooses the most affected side, the 2 mm tip of the Stylus should be used, which will produce the smallest possible resection from the lowest point of the tibial plateau. Using the ring you can make a micrometric vertical displacement controlled of the Tibial Resection Block.

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Once the final position of the Cutting Block is obtained, use two Pins to fix the Cutting Block: the Pins should be inserted through the "O" holes in the block.

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At this point you can release the Cutting Block by opening the front locking trigger. Push the Cutting Block through the Pin till the contact with the tibia and remove the Tibial Proximal Guide.

Use the Hook Extractor (300110380) and the Slide-Hammer (300110277) to extract the Proximal Guide.

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Then stabilize its position by inserting a third Pin in an oblique hole.

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Resect the distal femur using a 1.27 mm (0.050") saw.

Verify the correct planarity of the cut using the Angel Wing and, if necessary, refine the cut.

Then remove the Cutting Block by sliding it outwards and leaving the Pins for the next steps.

NB: remove the oblique Pins prior to repositioning or removing the Distal Cutting Block.

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[OPTION: EXTRAROTATION MEASUREMENT IN FLEXION]]

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After performing the tibial resection, it is possible to evaluate the extrafemoral rotation through appropriate modular spacer blocks. To this end, it is necessary to connect an extrarotation module, from 0 to 6 degrees, to restore the correct tibial thickness: tibial component and polyethylene insert.

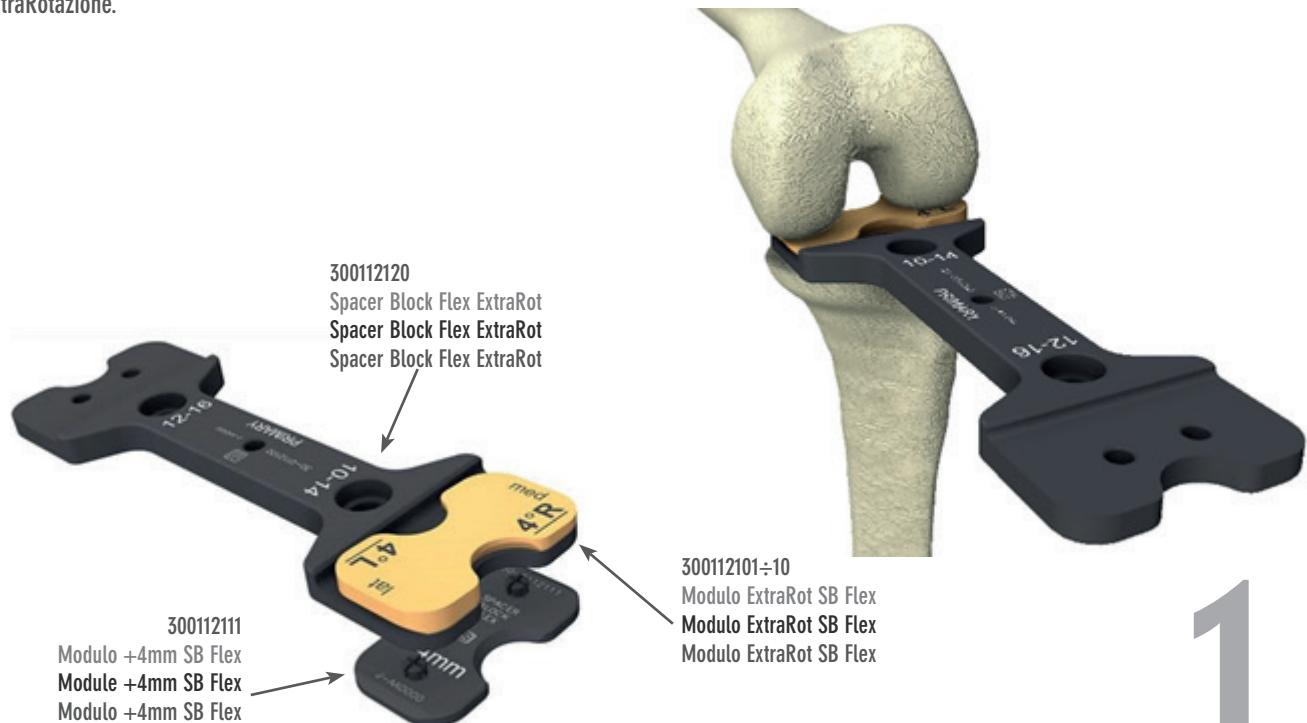
Select a Module ExtraRot SB Flex (300112101÷10), from 0 to 6 degrees, and connect it respecting the side of the leg to be operated, L (Left) or R (Right), to the Spacer Block Flex ExtraRot (300112120). The MED and LAT indications help in the correct positioning. Select the 10 or 12mm side according to the created space; if it is greater, place the Module +4mm SB Flex (300112111) in the lower part to simulate a space of 14 or 16mm.

Position the leg at 90 degrees of flexion. Insert the Spacer Block between the resected tibia and the still intact posterior femoral condyles. Check the ligament balancing and the extrarotation even trying different ExtraRotation Modules.

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[OPTION: MOBILE TRIAL REDUCTION AFTER TIBIAL PREPARATION]

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In the case of an implant with Mobile UC inserts, the trial reduction can be carried out after tibial preparation.

Proceed as described in the TIBIA: PREPARATION chapter, after which it will be possible to insert the Cone For Mobile Trial (300110222) in the conical slot in the Baseplate.

300110222
Cone Trial del Piatto Mobile
Cone For Mobile Trial
Cono Para Platillo Móvil De Prueba

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If Headed Pins were used to fixate the Baseplate, remove them.

Position the Trial Mobile Insert (300110401 ÷ 36) on the Baseplate by coupling the hole on the bottom surface with the peg of the Cone For Mobile Trial.

The size of the tibial insert corresponds to that of the femoral component even if Skinny femoral components are subsequently used.

Carry out the checks described in the TRIAL REDUCTION chapter (section 6).

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By using the trial inserts, assessment of the thickness and the most suitable polyethylene insert may also be carried out after implanting the definitive femoral and tibial components.

The tibial insert must be the same size as the femoral component, even when Skinny femoral components are used. The thickness should correspond to that of the Spacer Block used for extension and flexion gaps.

MOBILE INSERT

Connect the pre-selected Tibial Trial Insert (300110401 ÷ 36; 300111401 ÷ 300111436) and the Conical Adaptor (300110400) and mount them on the Trial Mobile Spacer (sizes 1-2: 300110390; sizes 3-4: 300110391; sizes 5-6: 300110392), which is chosen on the basis of the measurement of the femoral component, and makes it possible to restore the correct thickness of the insert and provide for protection of the articular surface of the definitive tibia. Insert the trial assembly in the definitive tibia.

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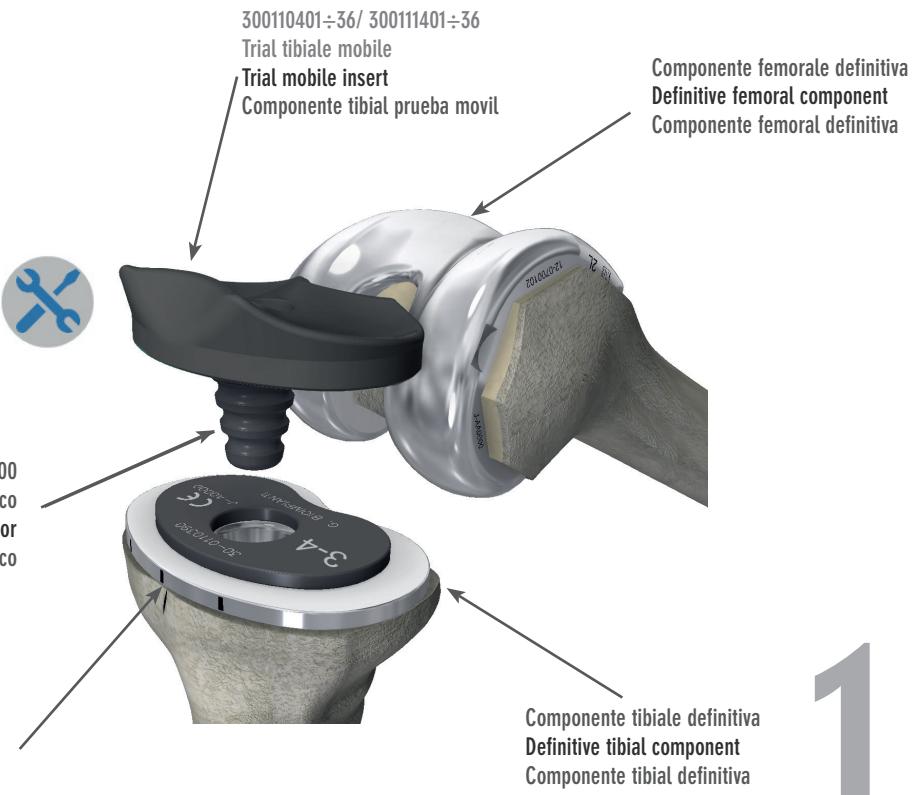
FIXED INSERT

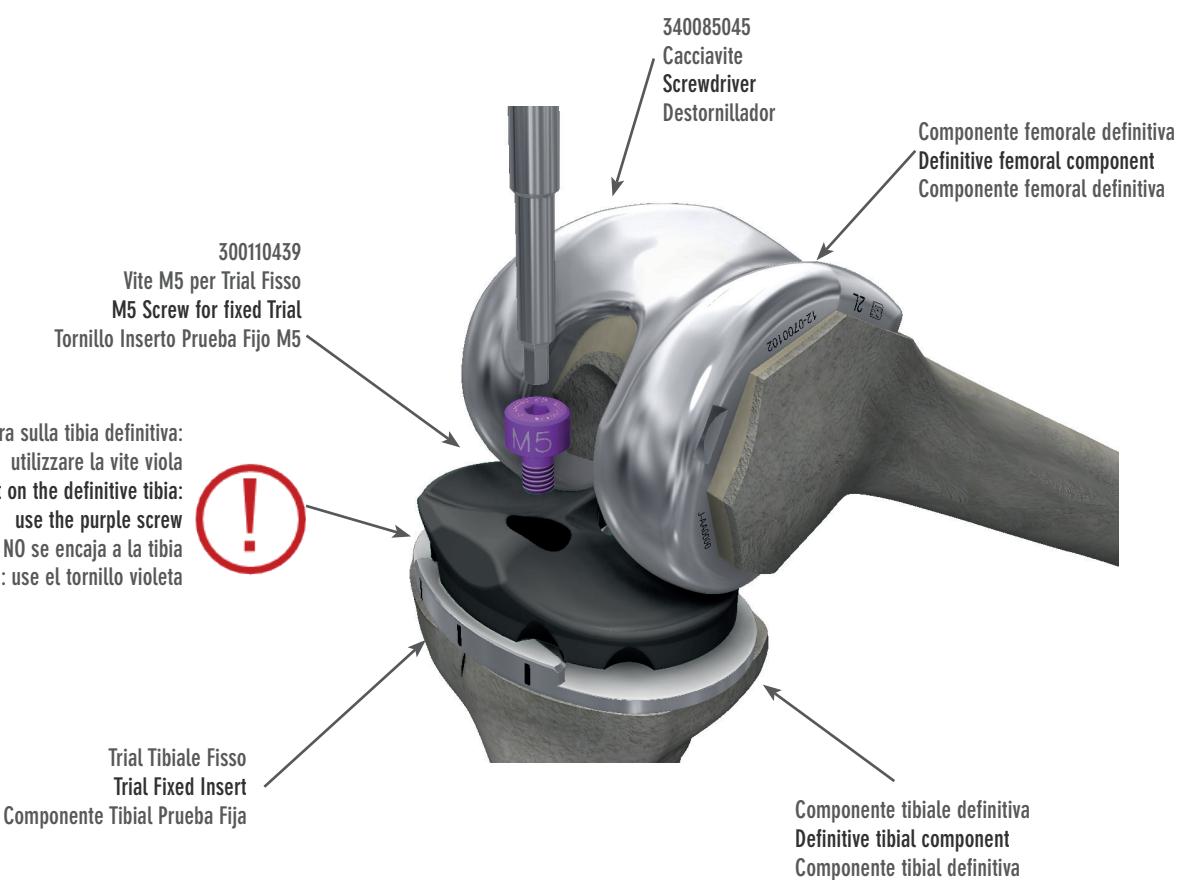
Choose between the options: Posterior Cruciate Ligament retaining CR (300110441 ÷ 76), Ultra Congruent UC (300110481 ÷ 516), Posterior Stabilized (300111301 ÷ 36) or Dynamic Congruence DC (300110521 ÷ 96) for the side of the leg being operated (L: Left; R: Right). Position the trial insert on the definitive tibia and anchor it in its housing with the M5 Screw for Fixed Trial (300110439) and Screwdriver (340085045). The screw is in the femoral instrument kit and coloured purple to distinguish it from the definitive one in the tibial component packets. The M5 Screw for Fixed Trial must not be used for the definitive articular insert.

After checking the stability of the joint in extension and flexion and choosing the most suitable implant, remove the trial insert and proceed to insert the definitive one as described in DEFINITIVE IMPLANTS, sections 3 and 4.

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If greater tibial stability is needed, the surgeon may use the additional tibial stems, which come in four lengths (30, 60, 100 and 140 mm) and diameters from 10 to 16 mm and whose dimensions are independent of the size of the tibial component.

When these additional stems are used, and especially the longer ones, it is advisable not to exceed in the posterior slope of the tibial resection, so as to avoid conflict with the anterior diaphyseal bone.

Select the Tibial Stem Reamer of the desired diameter (300110610÷16). Mount the Tibial Stem Reamer Guide (300110600) from the connection side and with the cylindrical body facing the reamer side. Slide it over the Reamer and engage it by the notch for the desired length.

Preparation may proceed in sequence following that of the tibial keel described in the TIBIA: PREPARATION chapter. Here too, if greater stability is needed, the Baseplate can be fixed using the Headed Pins (300110197).

Connect the Tibial Stem Reamer to the drill and move it through the Tibial Keel Guide (300110199) connected to the Baseplate as far as the stop.

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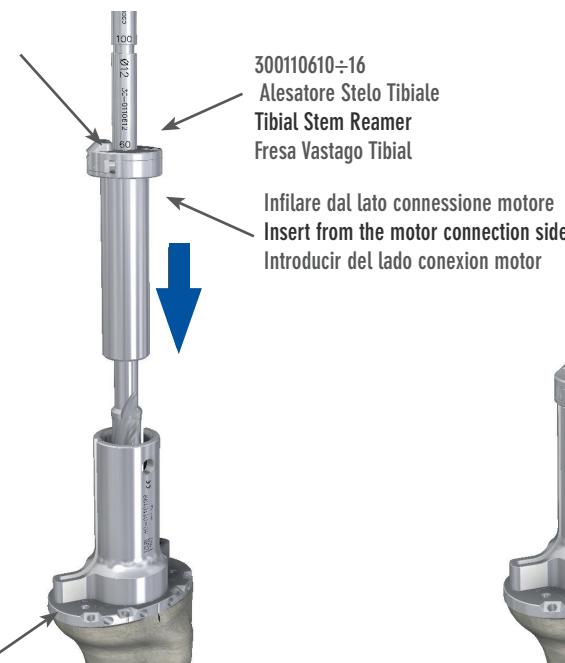
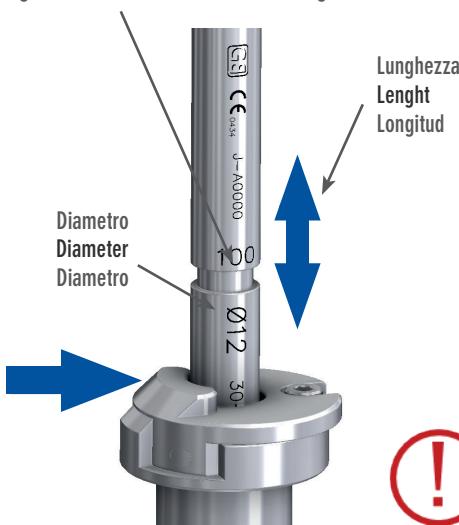
The system does not provide for a reduction with a trial stem, so proceed to remove all the instruments.

NB.: if desired, it is possible to proceed sequentially, both in the lengths and diameters of the Reamers.

Assembling the definitive components:
Open the definitive tibial component, fixed or mobile, and the additional tibial stem, as prepared. Unscrew the tibial stem's plug using the Screwdriver (340085045) and replace it with the additional tibial stem, making sure it's screwed in as far as possible. If bone cement is being used, it's possible to proceed with cementing up to the level of the connection. The additional stem is meant for use without cement.

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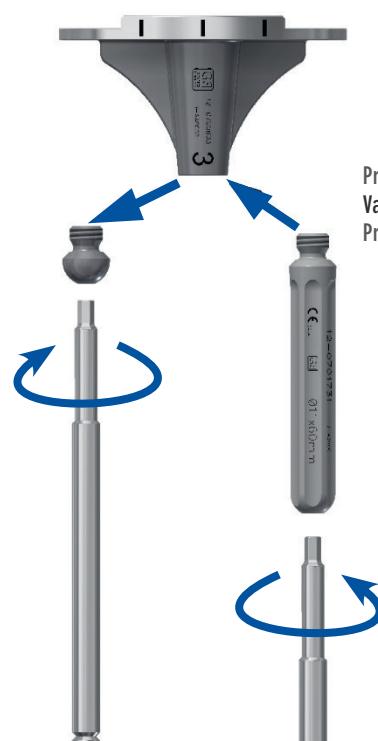
Remove any marginal osteophytes from the patella. If the thickness of the patella is to be measured, use the millimetric scale on the Patella Impaction Clamp (300110210).

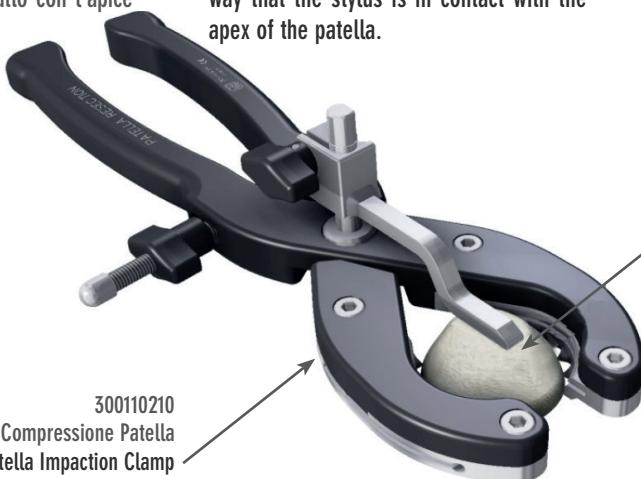
Position the Stylus of the Patella Resection Clamp (300110205) at a resection level of around 8-10mm. Grip the patella in such a way that the stylus is in contact with the apex of the patella.

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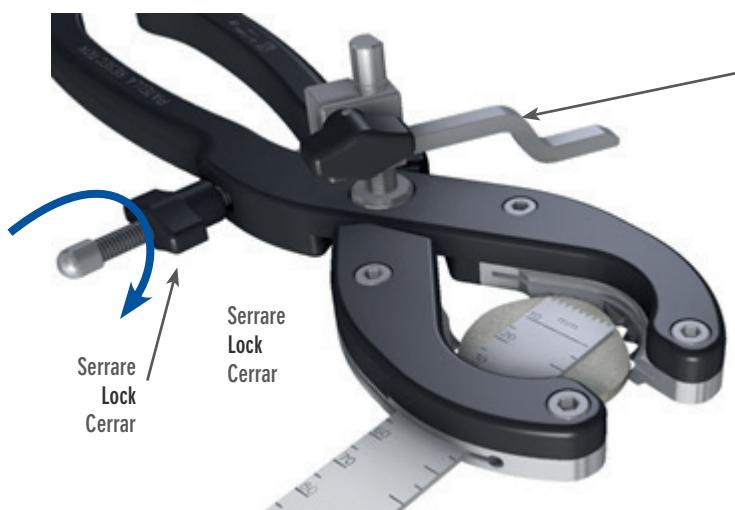
Evaluate the resection using the Angel Wing (300110139) in the lateral and medial slots and if necessary correct it by modifying the resection height of the stylus. Make particularly sure that the thickness of the residual patella is adequate and that the patellar ligament is secured.

Lock the Patella Resection Clamp, rotate the stylus and perform resection of the patella through the slots.

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Measure the patella diameter with the Sizing/Drilling Guide Patella (300110185) and choose the appropriate size.

The diameter of the patella component is independent of the size of the femoral component.

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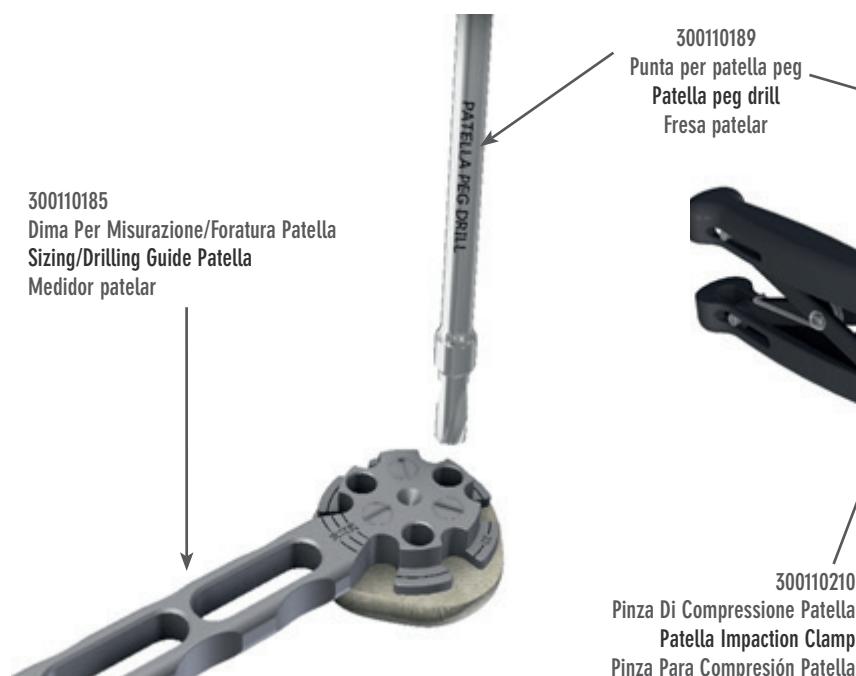
Drill three holes for the pegs of the patellar component using the Patella Peg Drill (300110189), drilling up to the stop. Alternatively, the Patella Impaction Clamp (300110210) may be used.

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
Position the Patella Trial Diameter 28, 32, 36 (300110186, 87, 88) of the chosen size on the patella and press lightly into the three holes. Check the patellar implant in the trial reduction.

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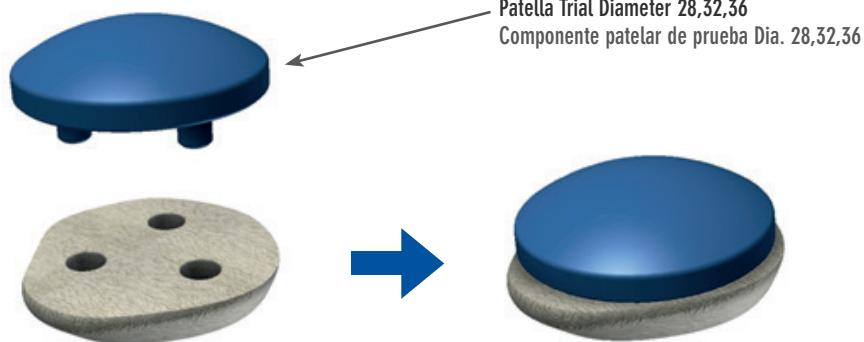Carefully choose the diameter of the definitive implant and then proceed to apply the cement. Align the three fixation pegs with the three holes drilled in the patella and press lightly to insert the component. Complete the positioning with the Patella Impaction Clamp (300110210), whose screw adjustment makes it possible to keep the patellar implant compressed until the bone cement has set definitively.

Make sure there is perfect adherence between component and resection and remove any excess cement.

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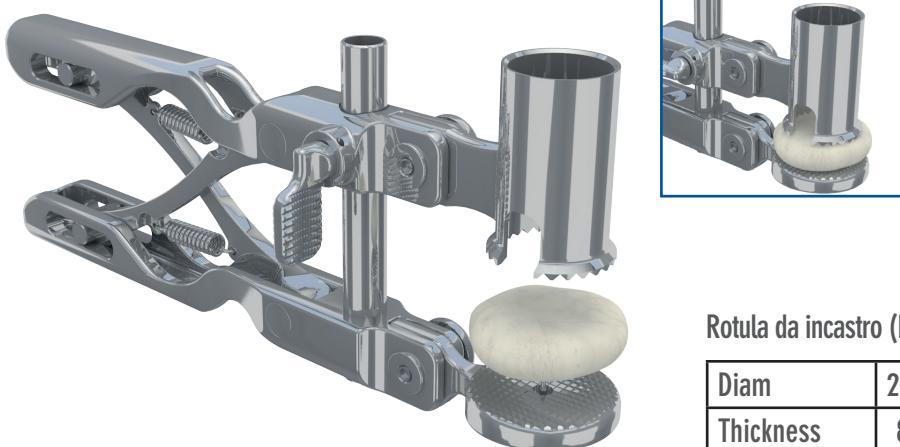
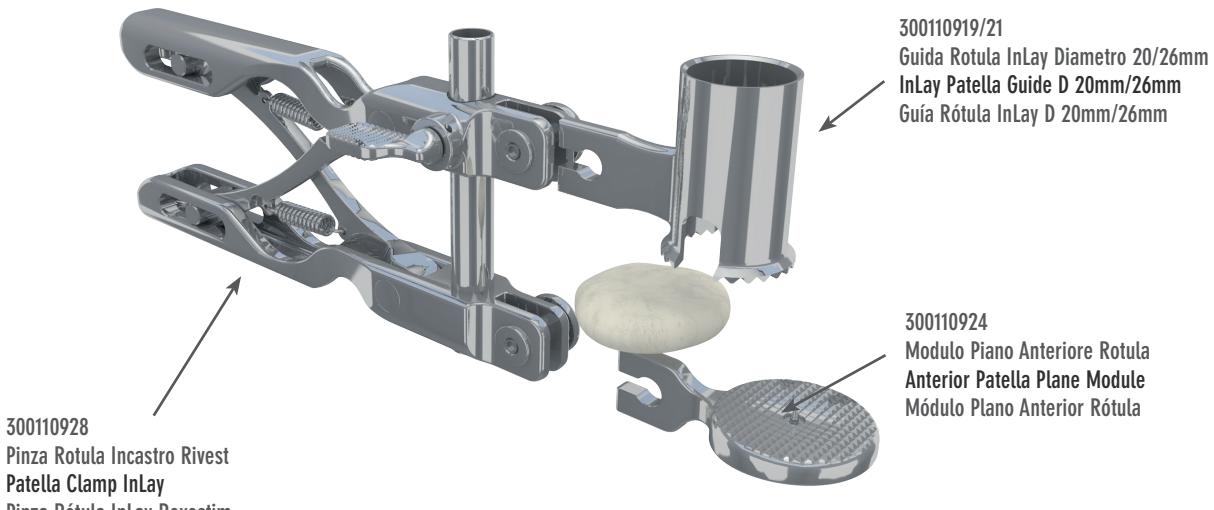
Select the diameter of the guide so that it is significantly within the patellar perimeter and center it on the central rim of the articular surface.

Coloque el K-MOD Módulo Plano Anterior Rótula (300110924) y el K-MOD Guía Rótula InLay D 20mm/26mm (300110919/21) en la K-MOD Pinza Rótula InLay Revestim (300110928).

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
Based on the chosen diameter, select the InLay Patella Reamer D 20mm/26mm (300110904/06) and hook it to the Universal Patella Reamer Rod (300110901). Insert it into the Guide until it comes into contact with the patellar apex. Place the Reamer Calibration Thickness (300110903) on the upper edge of the Guide, selecting an appropriate milling thickness. Place the Patella Ring Stop (300110902) on the Shims and lock it in place.

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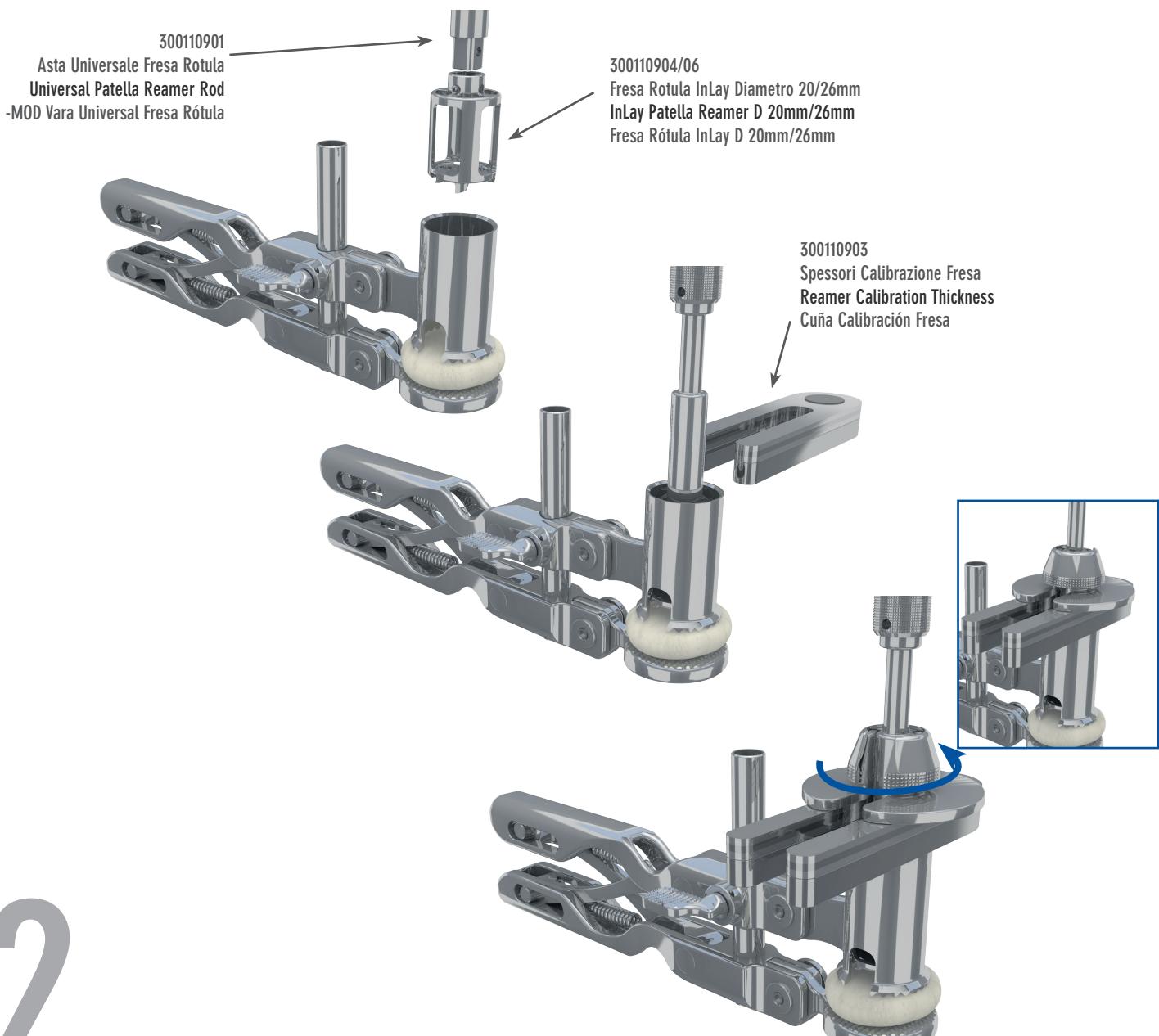
Rimuovere la Prova Rotula InLay.

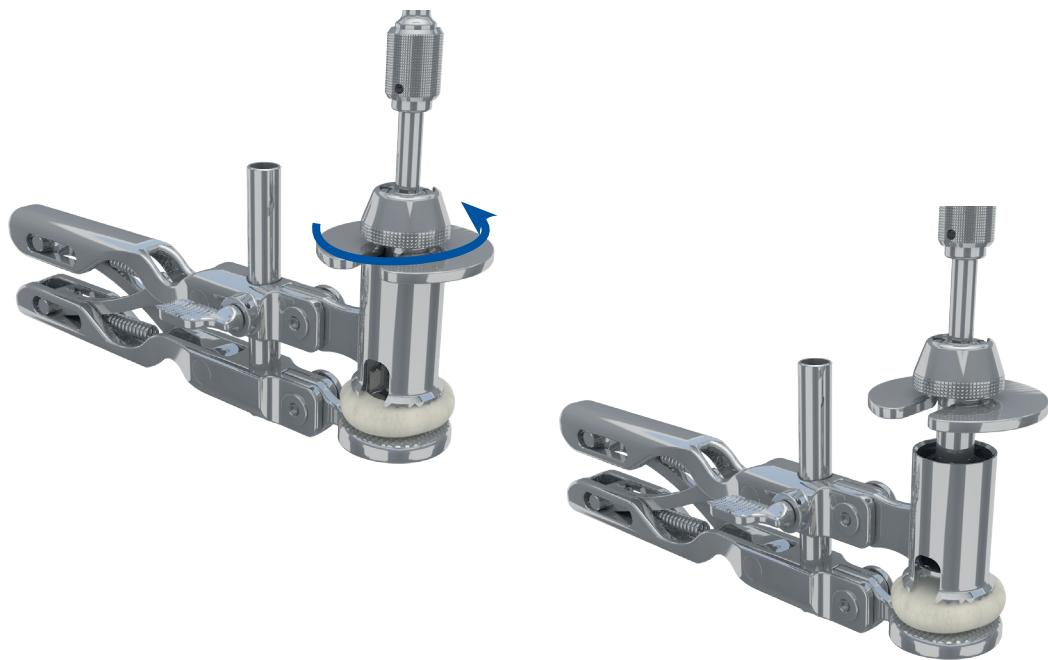Remove the Cutter Calibration Shims and proceed with milling until the Stop Ring comes into contact with the upper edge of the guide.

According to the chosen diameter, place the K-MOD InLay Patella Trial D 20mm/26mm (300110925/27) and perform the trial reduction.

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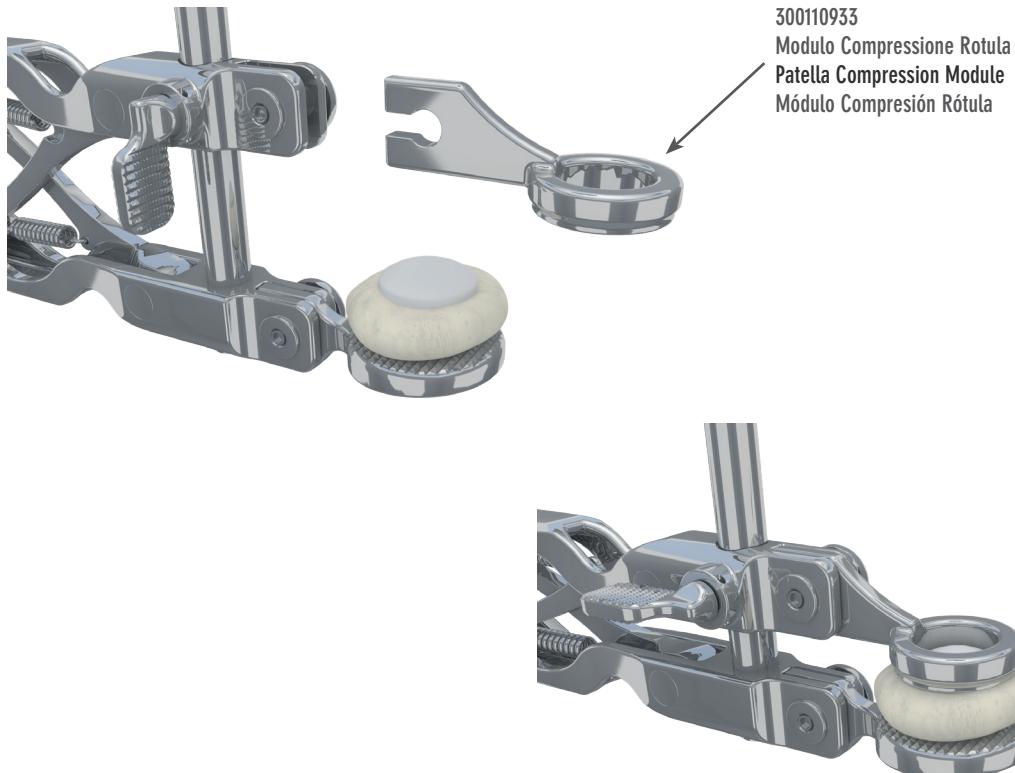
Select the definitive component based on the chosen diameter, proceed with the appropriate cementation, position it and fix it by tightening the forceps until the end of the cementation phase.

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K-MOD UNI REV instrument set is the solution in case of unicompartmental knee prosthesis revision.

K-MOD UNI REV through special wedges, allows to reconstruct the distal and posterior articular femoral lines in order to use K-MOD 3REF instrumentation.

Remove the failed femorotibial unicompartmental knee prosthesis.

To facilitate the operation it is possible to use the Flat, Pointed, Wedge Chisels (300110780/2) and the Femur Extractor (300114123).

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
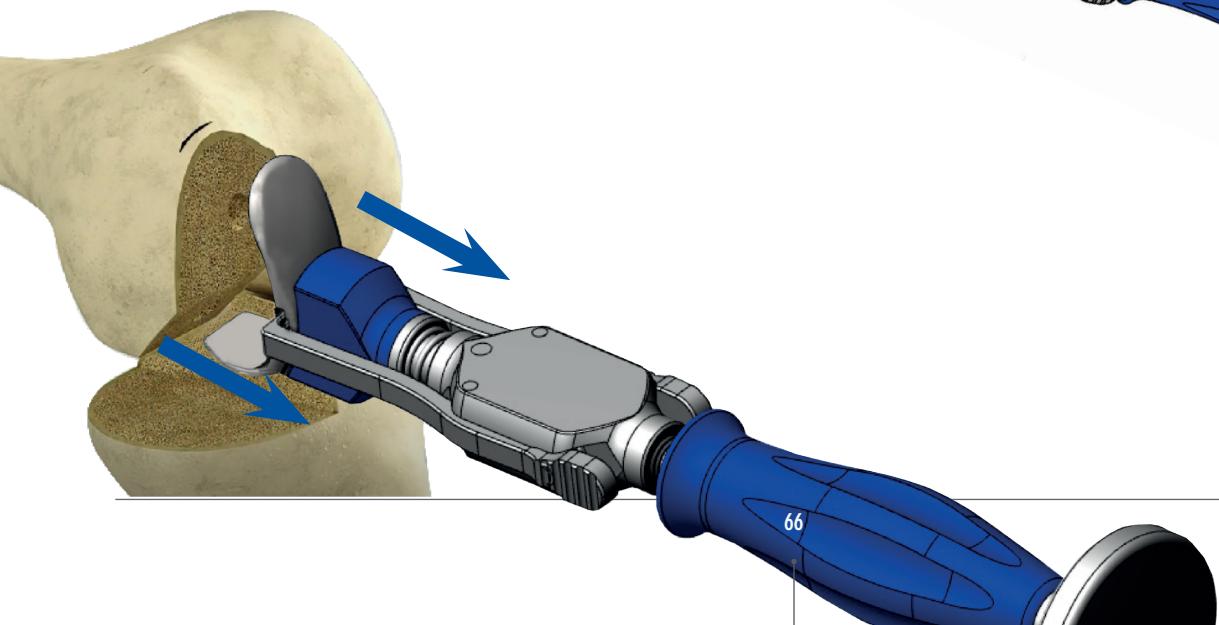
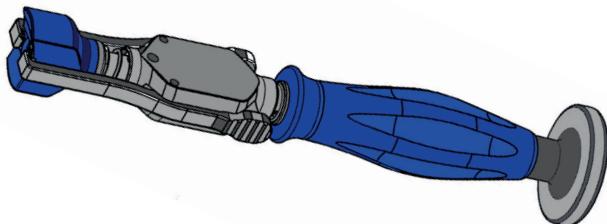
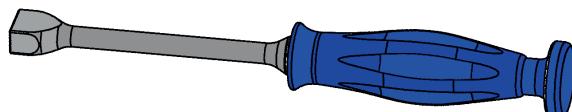
2

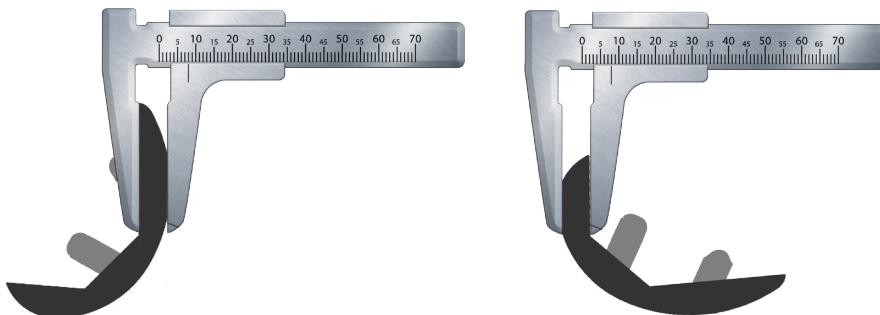Measure the distal and posterior thicknesses of the removed unicompartmental femoral component.

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Based on the measured distal thickness, select the corresponding K-MOD UNI REV Distal Thickness 4÷9mm (300110794÷9). The thickness simulates the missing condyle after removal of the prosthesis. It is possible to select the same value or a different one if you intend to vary the joint line compared to the previous one.

Assemble the chosen Distal Thickness on the ExtRot Femur Sizer (300110702) in the compartment of the removed prosthesis. Assemble the K-MOD 3REF Femoral Valgus Guide (300110701) with the FRONT indication facing upwards and finally the Distal Femur Cutting Block (300110159) with the DISTAL FEMUR indication visible upwards.

Proceed with the distal resection in accordance with the chapter of operating technique K-MOD FEMUR: DISTAL RESECTION.

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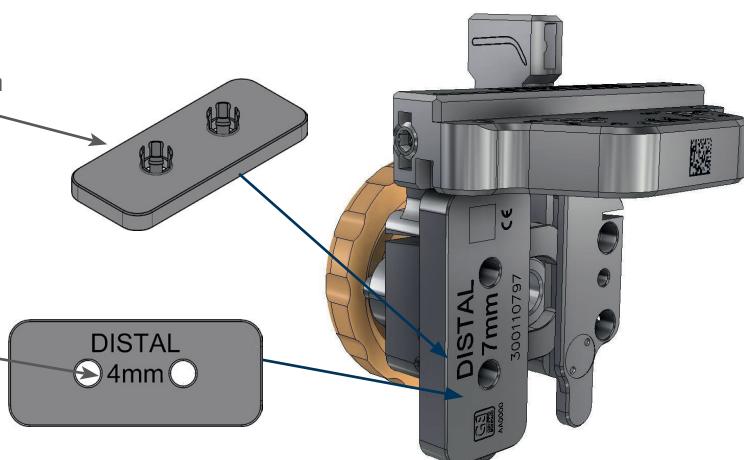
Based on the measured posterior thickness, select the corresponding K-MOD UNI REV Posterior Wedge 4÷9mm (300110787÷92). The thickness simulates the missing condyle after removal of the prosthesis. It is possible to select the same value or a different one if you intend to vary the joint line compared to the previous one.

Assemble the chosen Posterior Wedge on the Fem ExtraRotation Guide (300110703) in the compartment of the removed prosthesis. Assemble the ExtRot Femur Sizer (300110702), from which the Distal Wedge previously used has been removed, and finally insert the Femoral Stylus (300110167) into the Sizer.

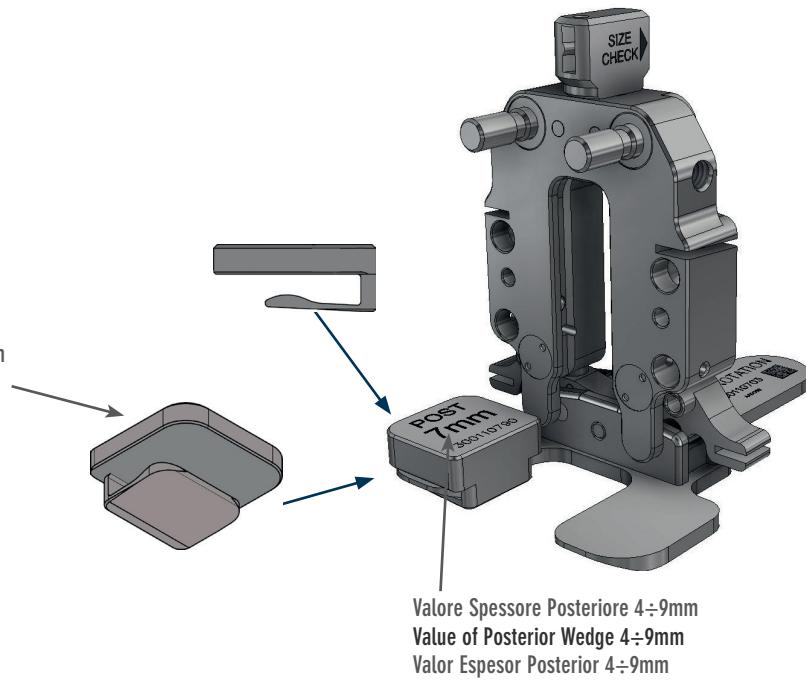
Proceed with the measurement, choice of external rotation and femoral reference in accordance with what is reported in the chapter of operating technique 3REF FEMUR: MEASUREMENT AND RESECTIONS 4IN1.

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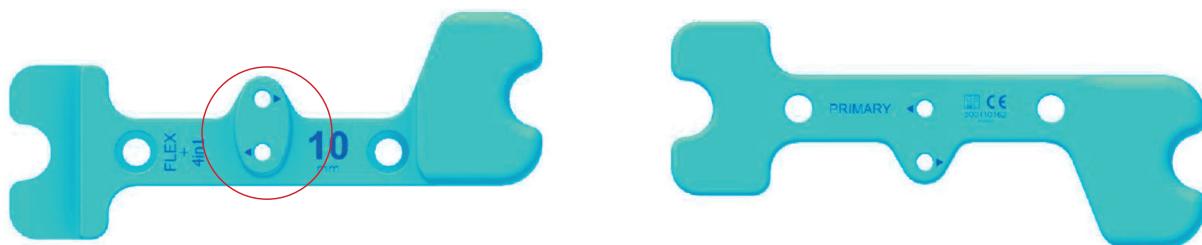
Peninsula (of increased height) with hole to correctly center the rod in extension

300110162 K-MOD Espaciador 10mm
 300110163 K-MOD Espaciador 12mm
 300110164 K-MOD Espaciador 14mm
 300110165 K-MOD Espaciador 16mm

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Possibility of inserting two oblique pins on both the medial and lateral sides

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300110377 K-MOD Jacobs Adaptor

Retention on the pin with a tab mechanism

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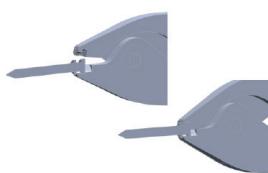
300110282 K-MOD Multipin Clamp

Hook for pin with and without head iand for insertion and extraction

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300110145 K-MOD IM Reamer

The IM hole entrance area has been enlarged to reduce constraint and facilitate channel search

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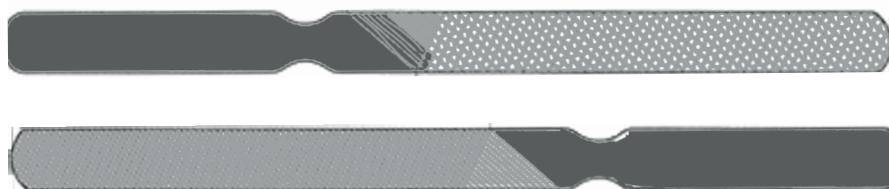
300110145 K-MOD Bone rasp

Flat rasp with:
- Fine grain side
- Coarse grain side

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Two holes on the femoral shield to stabilize the femoral component

300110223-28 K-MOD Femur de prueba 1-6 D
300110229-34 K-MOD Femur de prueba 1-6 Iz
300111201-06 K-MOD PS Femur de prueba 1-6 D
300111221-26 K-MOD PS Femur de prueba 1-6 Iz

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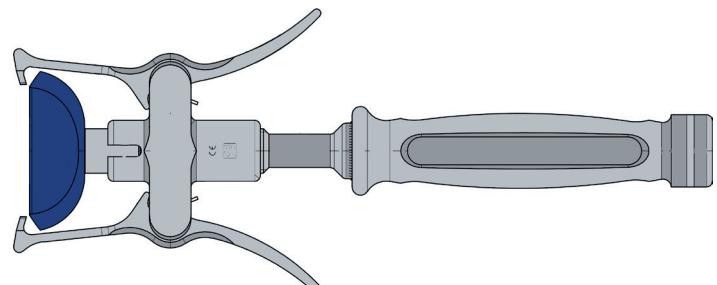
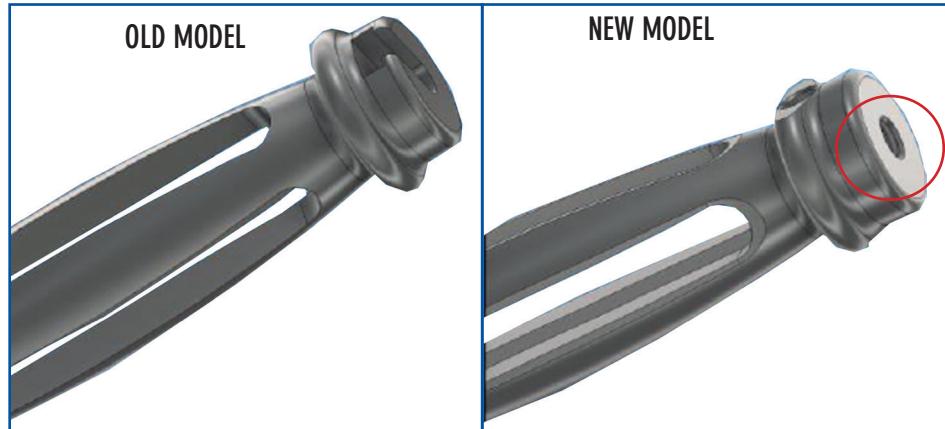
300110235 K-MOD Femoral Positioner

The terminal design has been modified with the addition of two holes for coupling with the hook and the striking mass to facilitate extraction operations

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
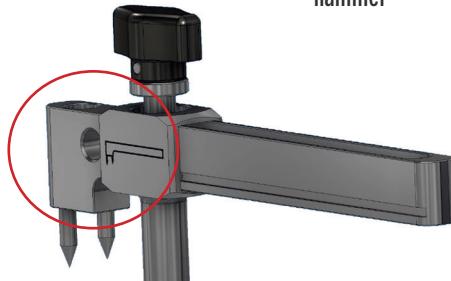
300110116 IM Tibial Adaptor

Double possibility for extraction with the hammer

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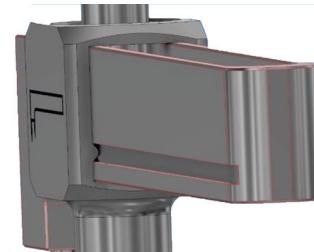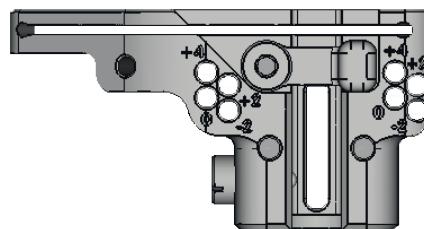
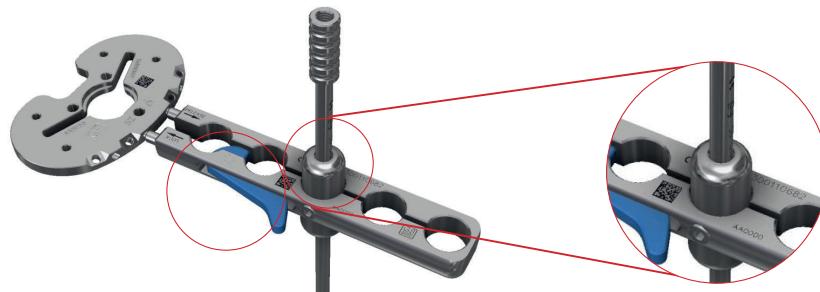
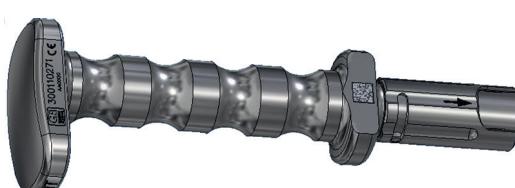
300110271 K-MOD Tibial Broach Handle


Easier extraction: increased front upper impact surface (ANT marking)

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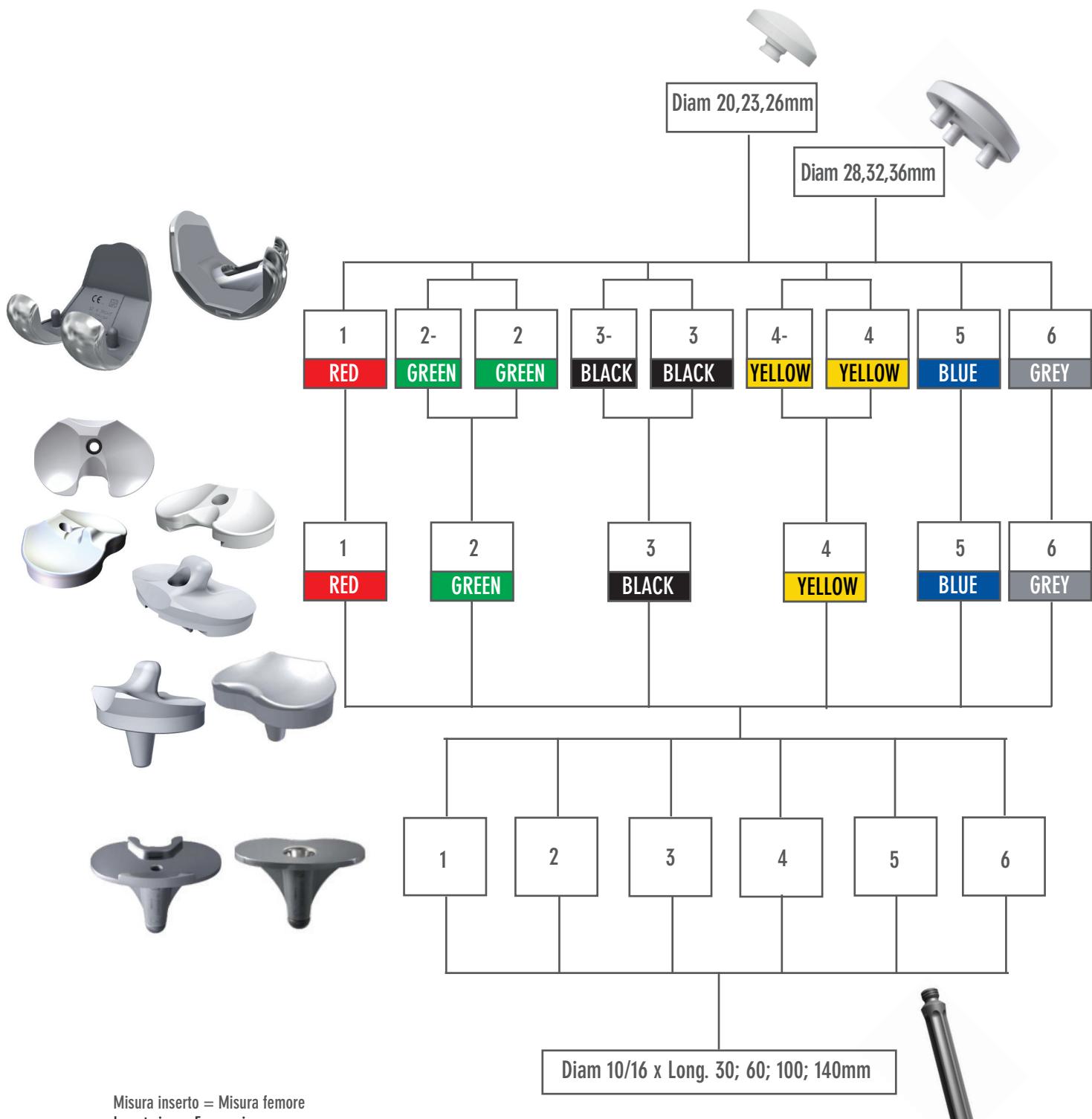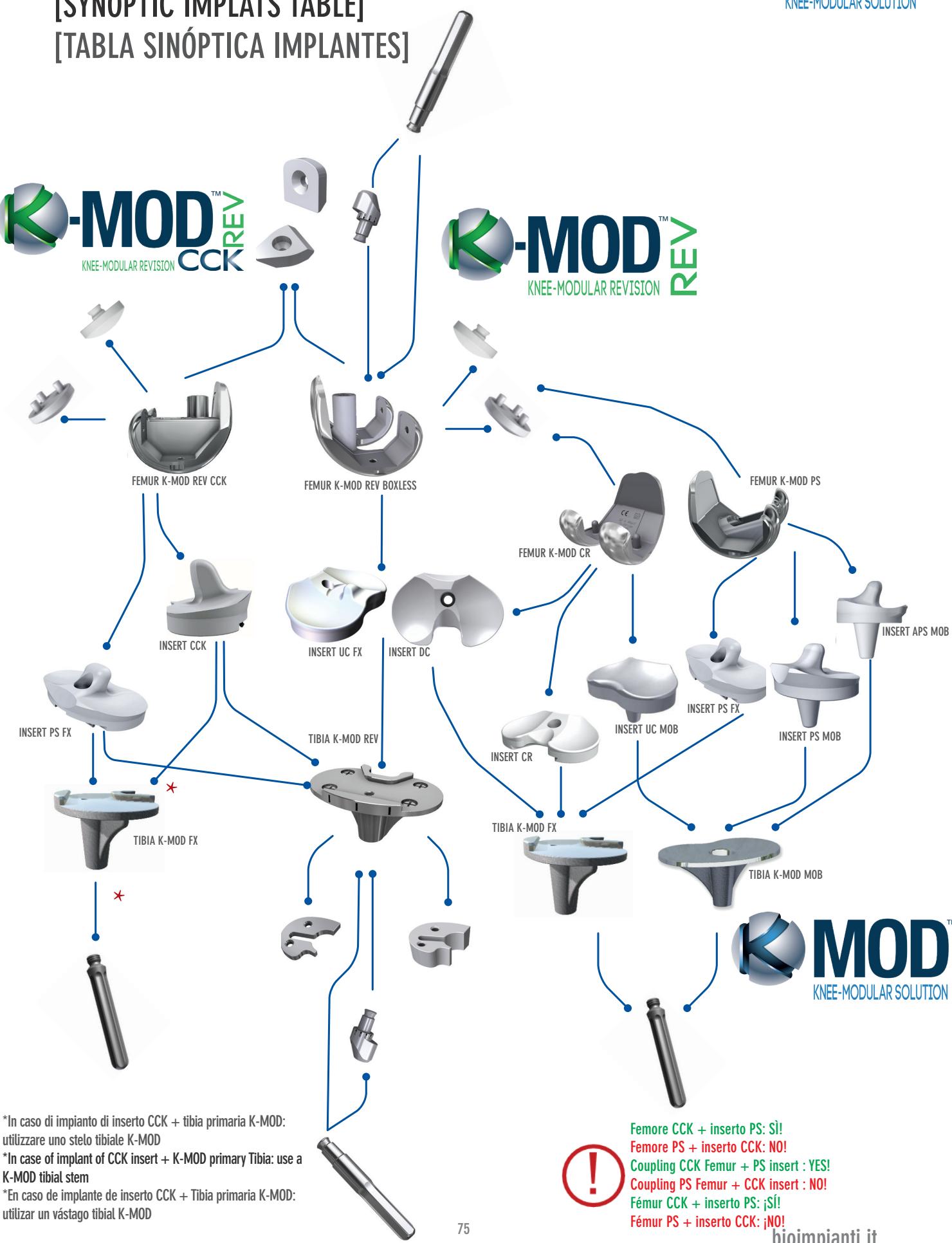
[FEMORE CEMENTATO ALLERGY]
[ALLERGY CEMENTED FEMUR]
[FEMUR CEMENTADO ALLERGY]

Ref.	DESCRIZIONE	DESCRIPTION	DESCRIPCIÓN	ALLERGY
120701101	K-MOD Femore cementato TG.1 Sx.	K-MOD Cemented Fem. S.1 L	K-MOD Fémur Cementado T1 Izq	120701101A
120701102	K-MOD Femore cementato TG.2 Sx.	K-MOD Cemented Fem. S.2 L	K-MOD Fémur Cementado T2 Izq	120701102A
120701103	K-MOD Femore cementato TG.3 Sx.	K-MOD Cemented Fem. S.3 L	K-MOD Fémur Cementado T3 Izq	120701103A
120701104	K-MOD Femore cementato TG.4 Sx.	K-MOD Cemented Fem. S.4 L	K-MOD Fémur Cementado T4 Izq	120701104A
120701105	K-MOD Femore cementato TG.5 Sx.	K-MOD Cemented Fem. S.5 L	K-MOD Fémur Cementado T5 Izq	120701105A
120701106	K-MOD Femore cementato TG.6 Sx.	K-MOD Cemented Fem. S.6 L	K-MOD Fémur Cementado T6 Izq	120701106A
120701112	K-MOD Fem. Cementato Skinny TG.2- Sx	K-MOD Cemented Fem. Skinny S.2- L	K-MOD Fémur Cem Skinny T2- Izq	120701112A
120701113	K-MOD Fem. Cementato Skinny TG.3- Sx	K-MOD Cemented Fem. Skinny S.3- L	K-MOD Fémur Cem Skinny T3- Izq	120701113A
120701114	K-MOD Fem. Cementato Skinny TG.4- Sx	K-MOD Cemented Fem. Skinny S.4- L	K-MOD Fémur Cem Skinny T4- Izq	120701114A
120701201	K-MOD Femore cementato TG.1 Dx.	K-MOD Cemented Fem. S.1 R	K-MOD Fémur Cementado T1 Der	120701201A
120701202	K-MOD Femore cementato TG.2 Dx.	K-MOD Cemented Fem. S.2 R	K-MOD Fémur Cementado T2 Der	120701202A
120701203	K-MOD Femore cementato TG.3 Dx.	K-MOD Cemented Fem. S.3 R	K-MOD Fémur Cementado T3 Der	120701203A
120701204	K-MOD Femore cementato TG.4 Dx.	K-MOD Cemented Fem. S.4 R	K-MOD Fémur Cementado T4 Der	120701204A
120701205	K-MOD Femore cementato TG.5 Dx.	K-MOD Cemented Fem. S.5 R	K-MOD Fémur Cementado T5 Der	120701205A
120701206	K-MOD Femore cementato TG.6 Dx.	K-MOD Cemented Fem. S.6 R	K-MOD Fémur Cementado T6 Der	120701206A
120701212	K-MOD Fem. Cementato Skinny TG.2- Dx	K-MOD Cemented Fem. Skinny S.2- R	K-MOD Fémur Cem Skinny T2- Der	120701212A
120701213	K-MOD Fem. Cementato Skinny TG.3- Dx	K-MOD Cemented Fem. Skinny S.3- R	K-MOD Fémur Cem Skinny T3- Der	120701213A
120701214	K-MOD Fem. Cementato Skinny TG.4- Dx	K-MOD Cemented Fem. Skinny S.4- R	K-MOD Fémur Cem Skinny T4- Der	120701214A

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[TIBIA MOBILE CEMENTATA ALLERGY]
[MOBILE ALLERGY CEMENTED TIBIA]
[TIBIA MOVIL NO CEMENTADA ALLERGY]

[TIBIA MOBILE NON CEMENTATA]
[MOBILE UNCEMENTED TIBIA]
[TIBIA MOVIL NO CEMENTADA]

Ref.	DESCRIZIONE	DESCRIPTION	DESCRIPCIÓN	ALLERGY
120700501	K-MOD Tibia Mobile Non Cem. TG.1	K-MOD UnCemented Mobile Tibia S1	K-MOD Tibia Movil No Cementada T1	120700501A
120700502	K-MOD Tibia Mobile Non Cem. TG.2	K-MOD UnCemented Mobile Tibia S2	K-MOD Tibia Movil No Cementada T2	120700502A
120700503	K-MOD Tibia Mobile Non Cem. TG.3	K-MOD UnCemented Mobile Tibia S3	K-MOD Tibia Movil No Cementada T3	120700503A
120700504	K-MOD Tibia Mobile Non Cem. TG.4	K-MOD UnCemented Mobile Tibia S4	K-MOD Tibia Movil No Cementada T4	120700504A
120700505	K-MOD Tibia Mobile Non Cem. TG.5	K-MOD UnCemented Mobile Tibia S5	K-MOD Tibia Movil No Cementada T5	120700505A
120700506	K-MOD Tibia Mobile Non Cem. TG.6	K-MOD UnCemented Mobile Tibia S6	K-MOD Tibia Movil No Cementada T6	120700506A

[TIBIA MOBILE NON CEMENTATA ALLERGY]
[MOBILE ALLERGY UNCEMENTED TIBIA]
[TIBIA MOVIL NO CEMENTADA ALLERGY]

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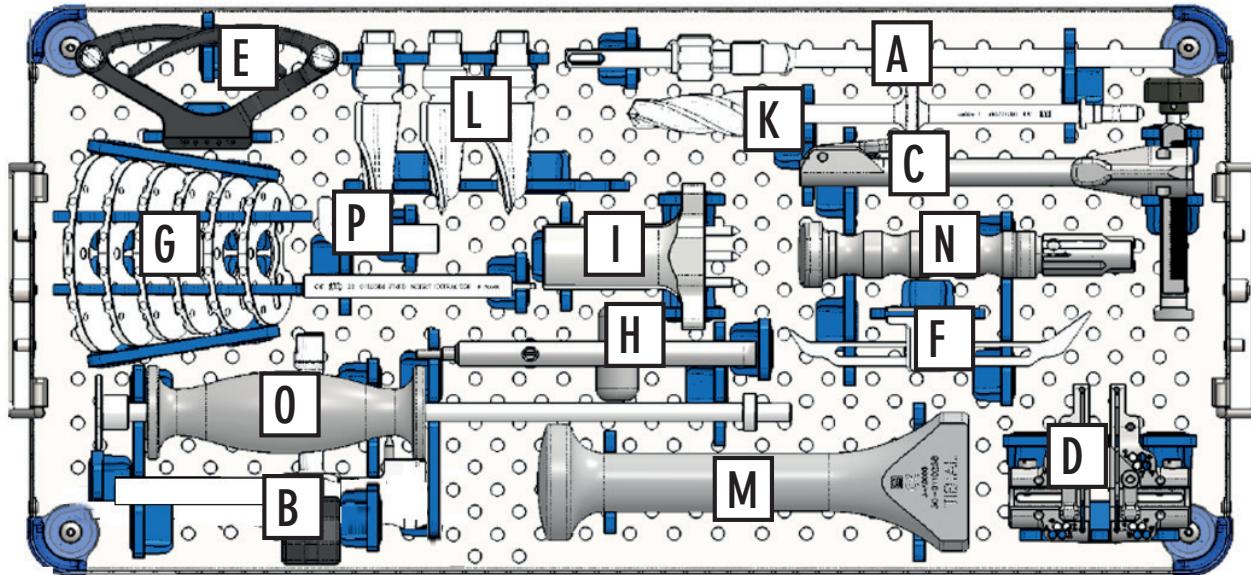
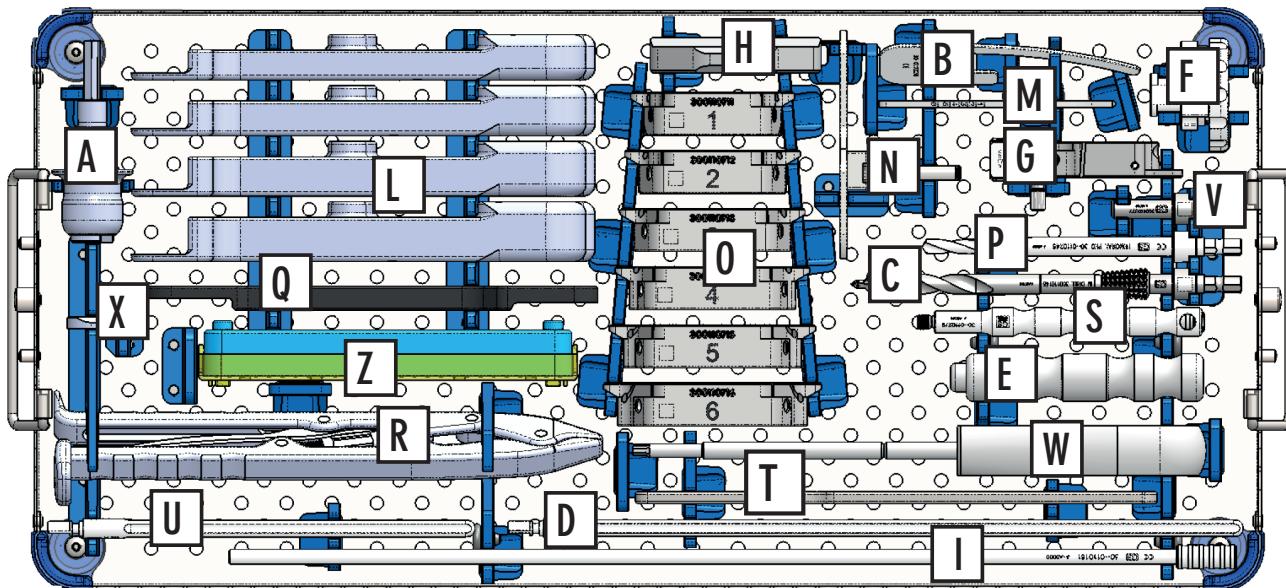
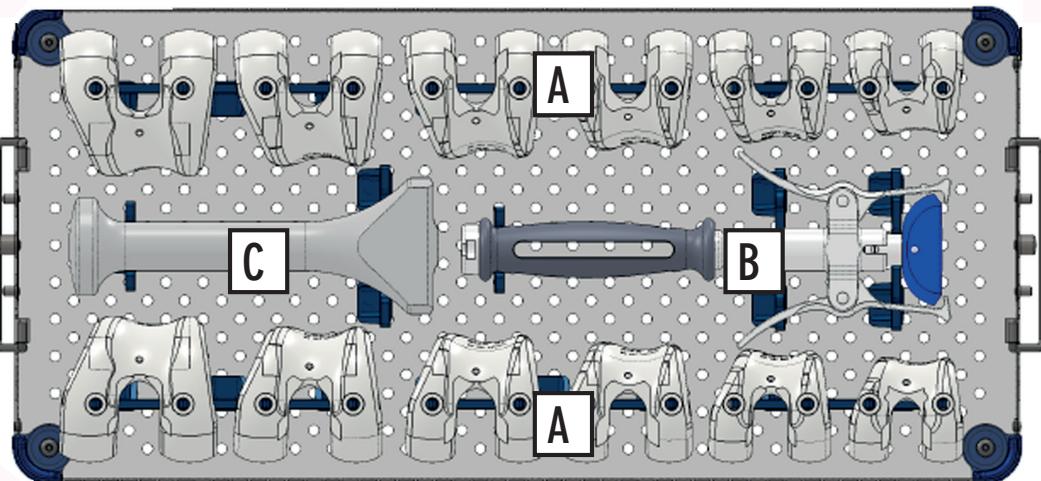
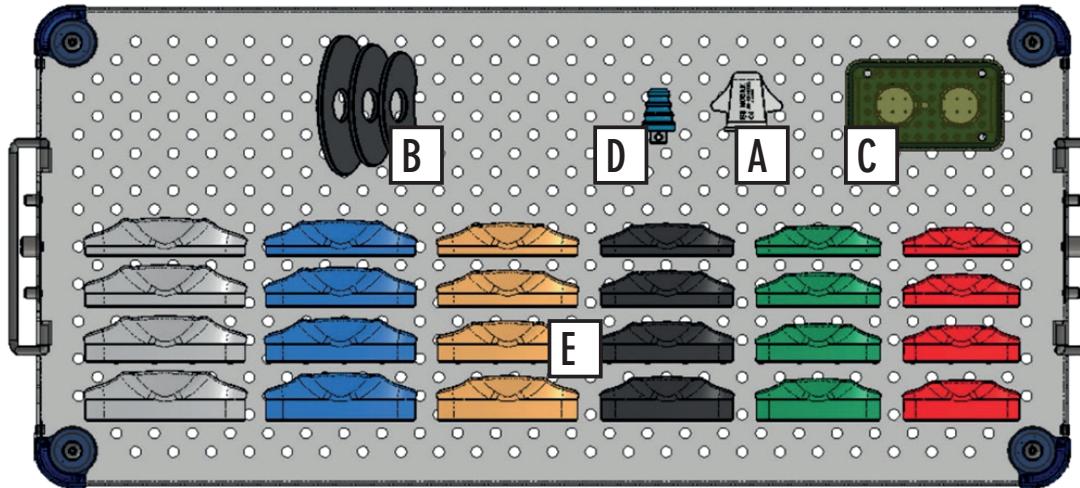
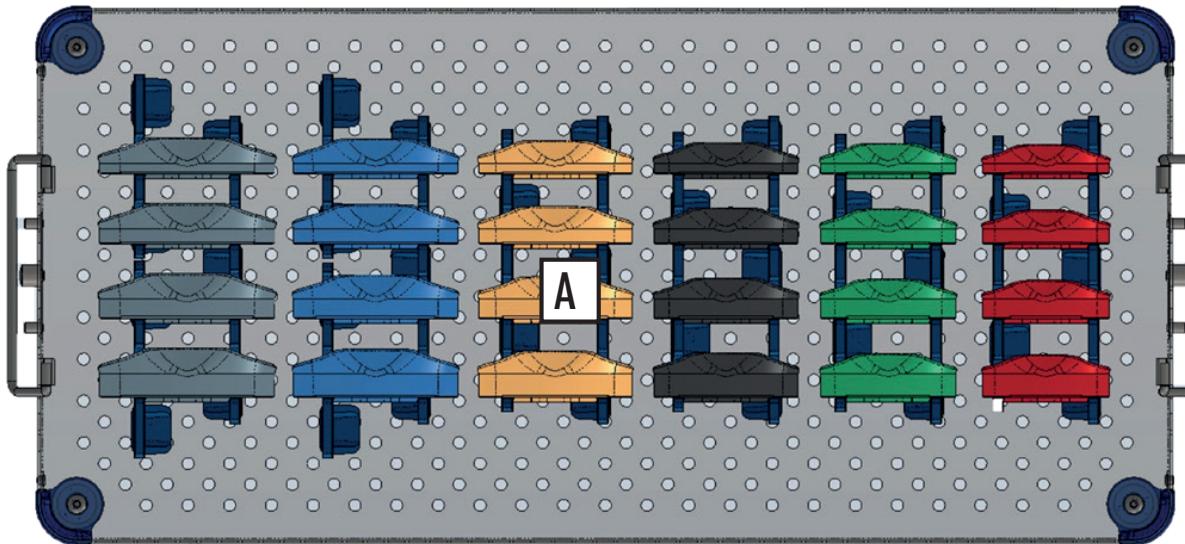
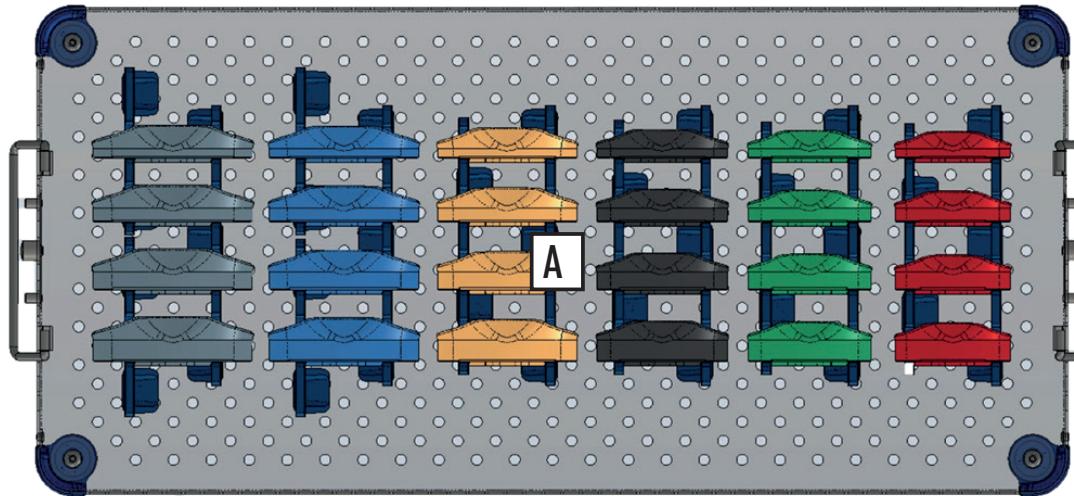
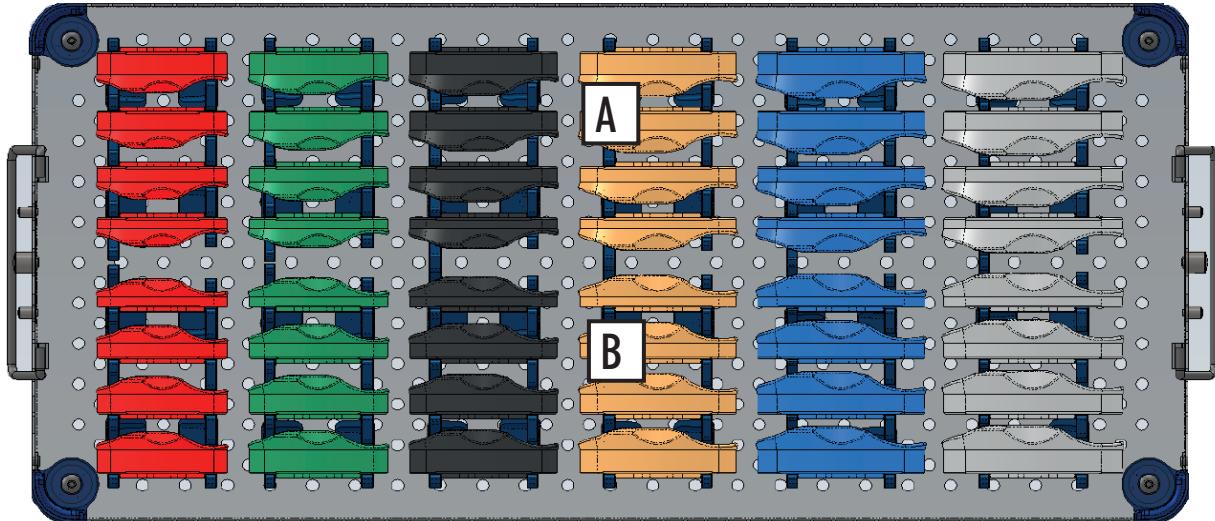
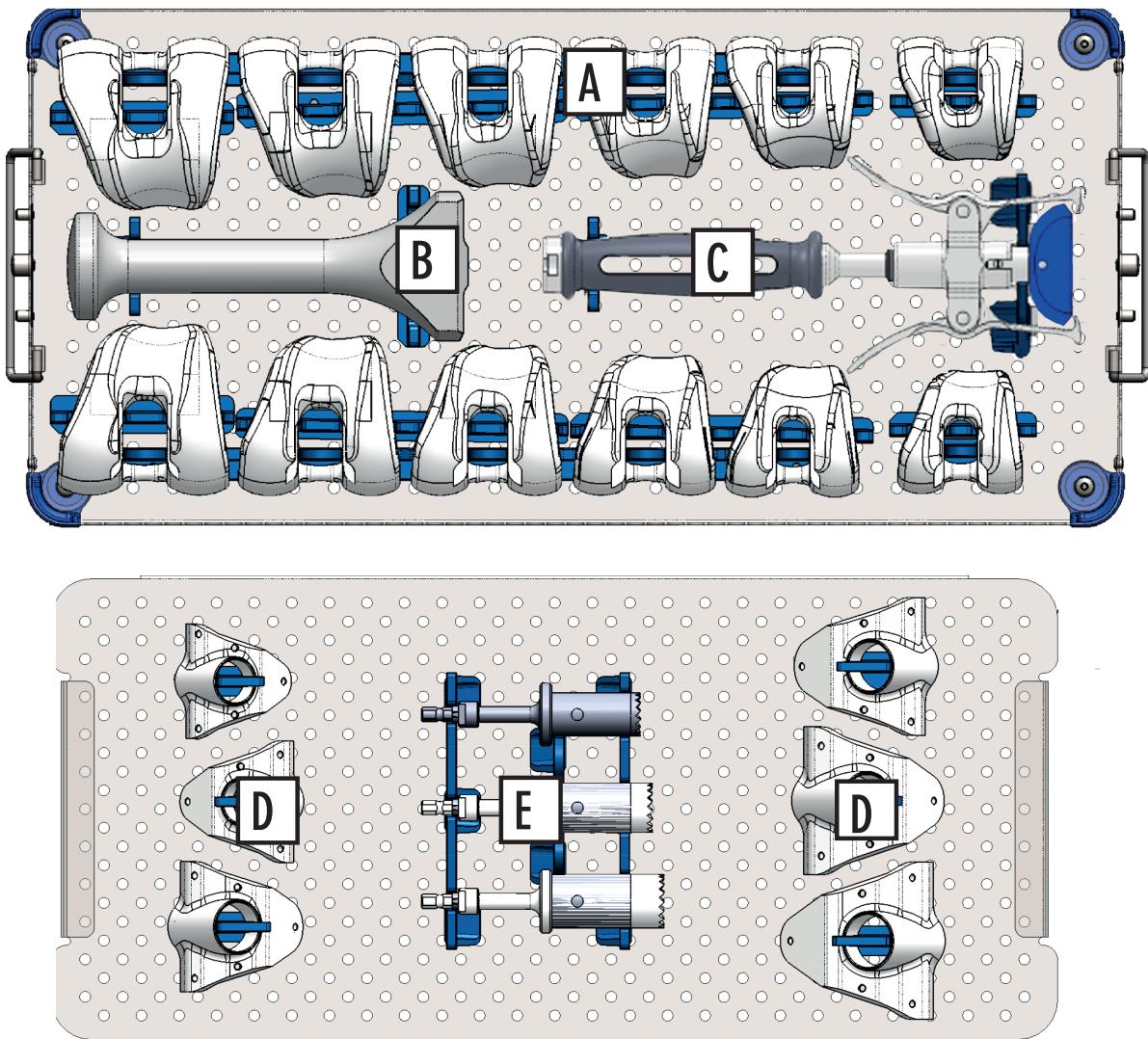
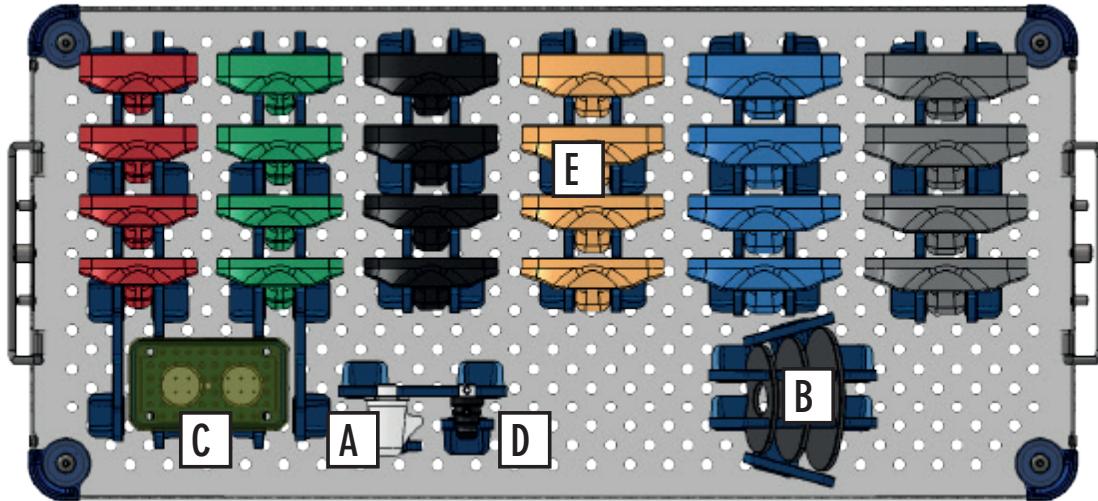
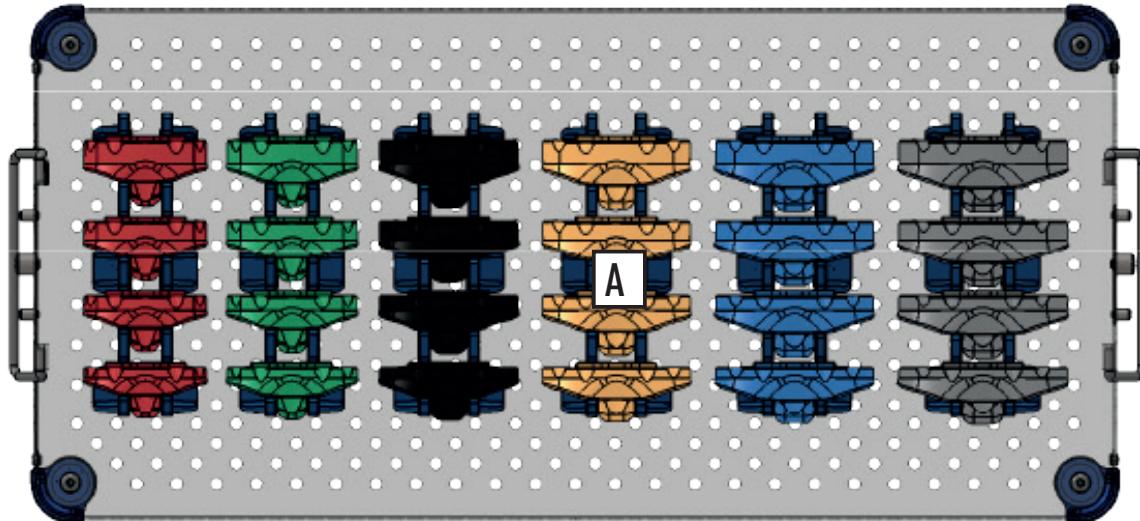
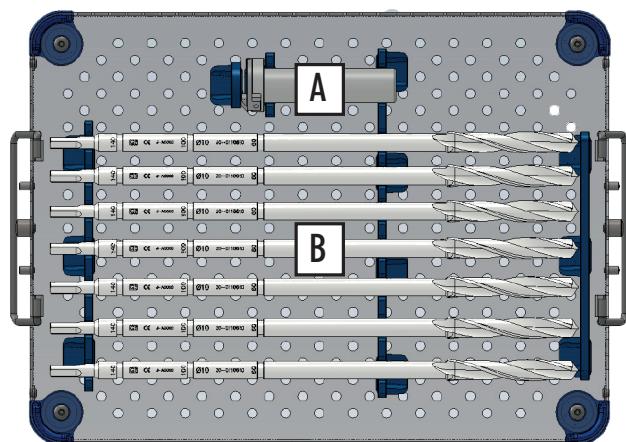
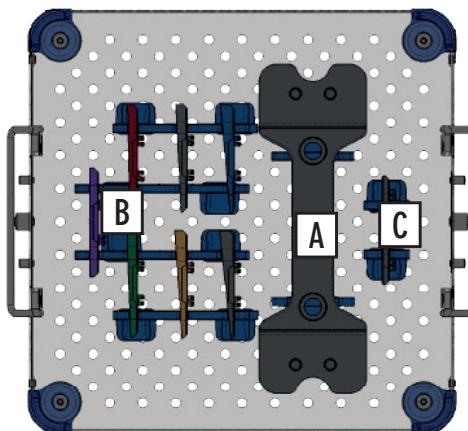
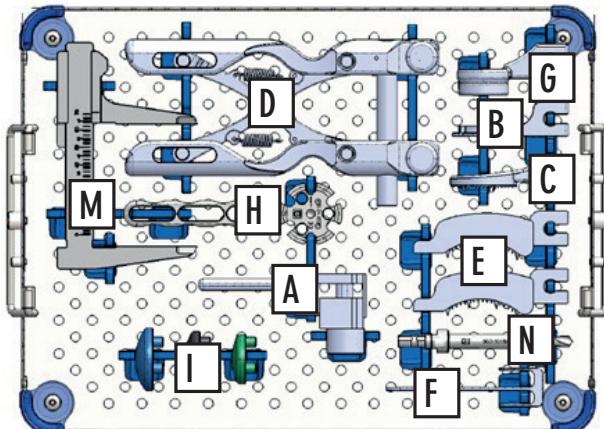
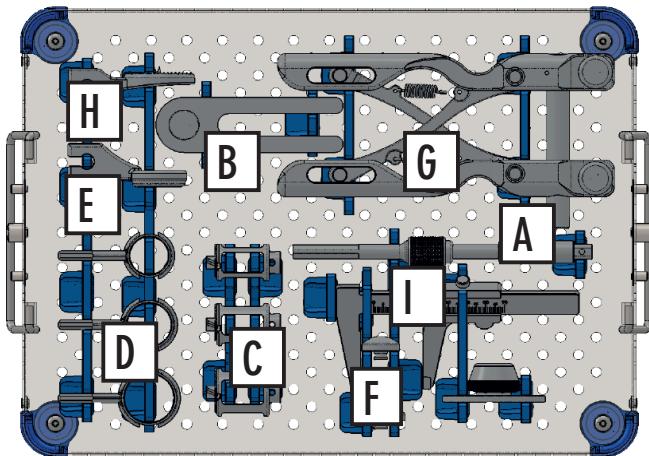
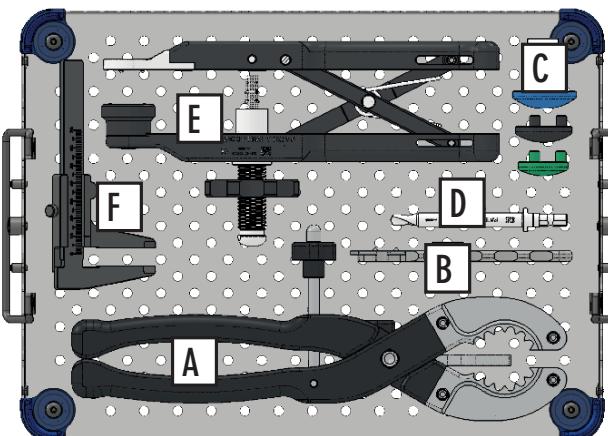
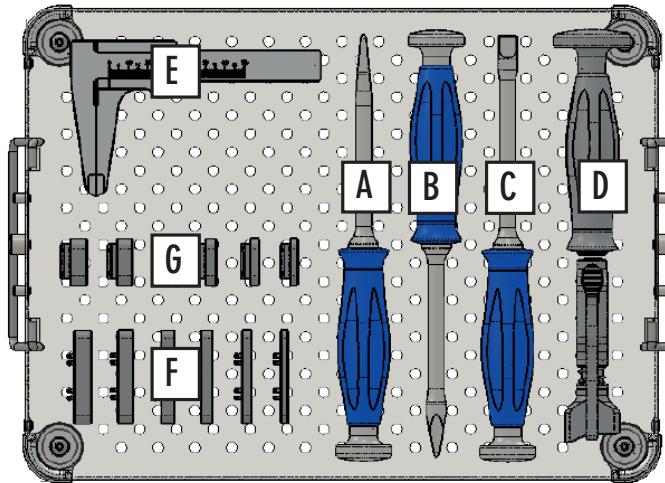
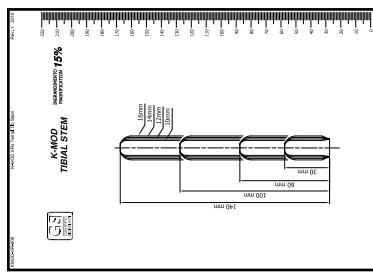
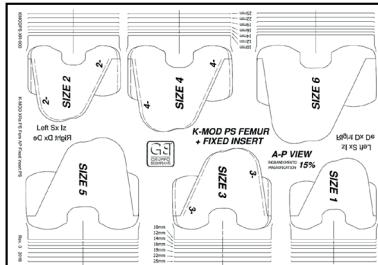
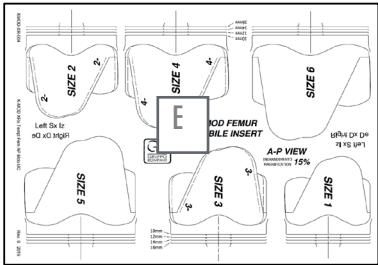
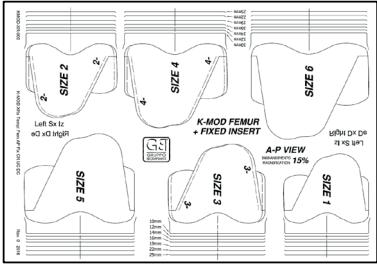
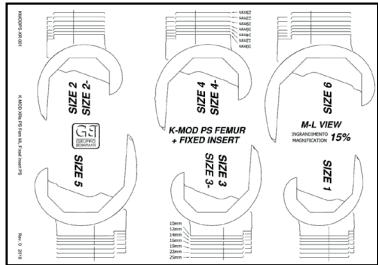
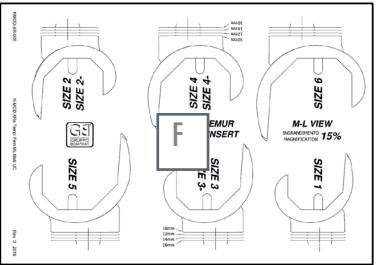
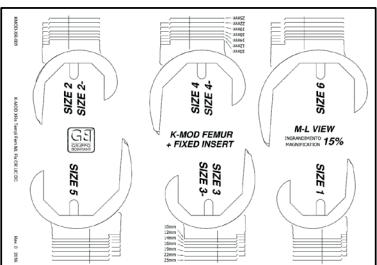
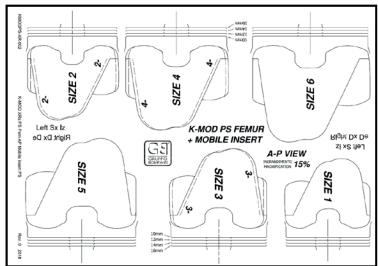
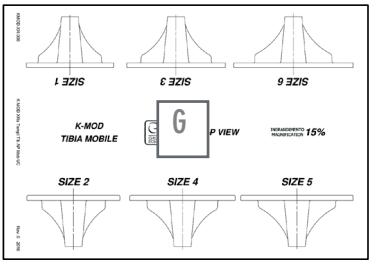
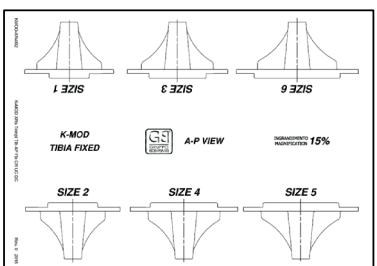
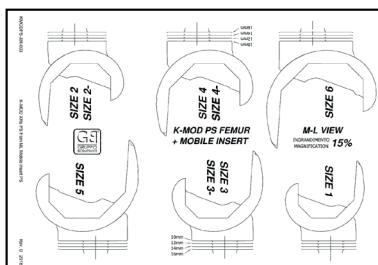
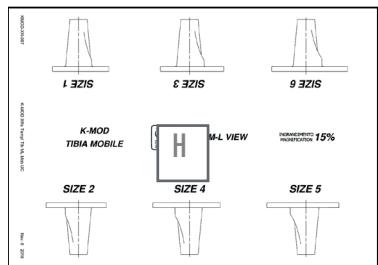
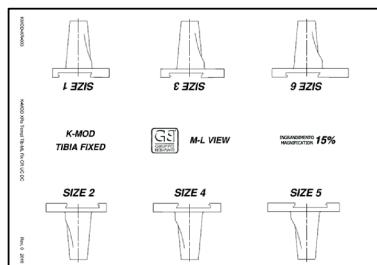
FIND OUT MORE ABOUT THE PRIMARY KNEE PROSTHESIS SYSTEM

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3REF Operating technique video

EN

Use the QR-Code to display the video that simulates the operating technique

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IFU

EN

Use the QR-Code to view the instruction for use

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The informations in this document refers to the basic system configuration and sizes.

To check the complete range, refer to the relative technical data sheet.

Some codes / products on request, developed for specific markets, may not be in immediate availability.

Check availability with your local GRUPPO BIOIMPIANTI distributor.

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Via Liguria 28, 20068 Peschiera Borromeo (Milan) Italy

Tel. +39 02 51650371 - Fax +39 02 51650393

info@bioimpianti.it

infofrance@bioimpianti.it

bioimpianti.it